ZimmerTerminal Flamenco, Best.-Nr. 77 0520 00

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RoomTerminal Flamenco, order no. 77 0520 00

Terminal for use in patient rooms, duty rooms, and function rooms, without speech communication. Red call button, blue alarm button, green presence button (staff 1), yellow presence button (staff 2) and 4 function keys as soft keys.

Control and monitoring of all room functions in compliance with DIN VDE 0834. Connection to the group bus (OSYnet) and the room bus (RAN).

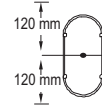
NOTE! The complete installation of the system is described in the Technical Manual.

CAUTION! The printed circuit board includes electrostatic sensitive components. Avoid touching.

Back box installation


When installing the back box consider the space required for the RoomTerminal. Measuring from the centre of the box, the available space must be at least 120 mm above and below.

A Mounting the pattress

1. Insert the connection cables through the outlet in the pattress [2].
2. Fix the pattress [2] using the back box screws [1] to the back box [3].

NOTE! If mounting on a back box is not possible, the RoomTerminal may be fixed to the wall with dowels and screws using the same holes in the pattress.

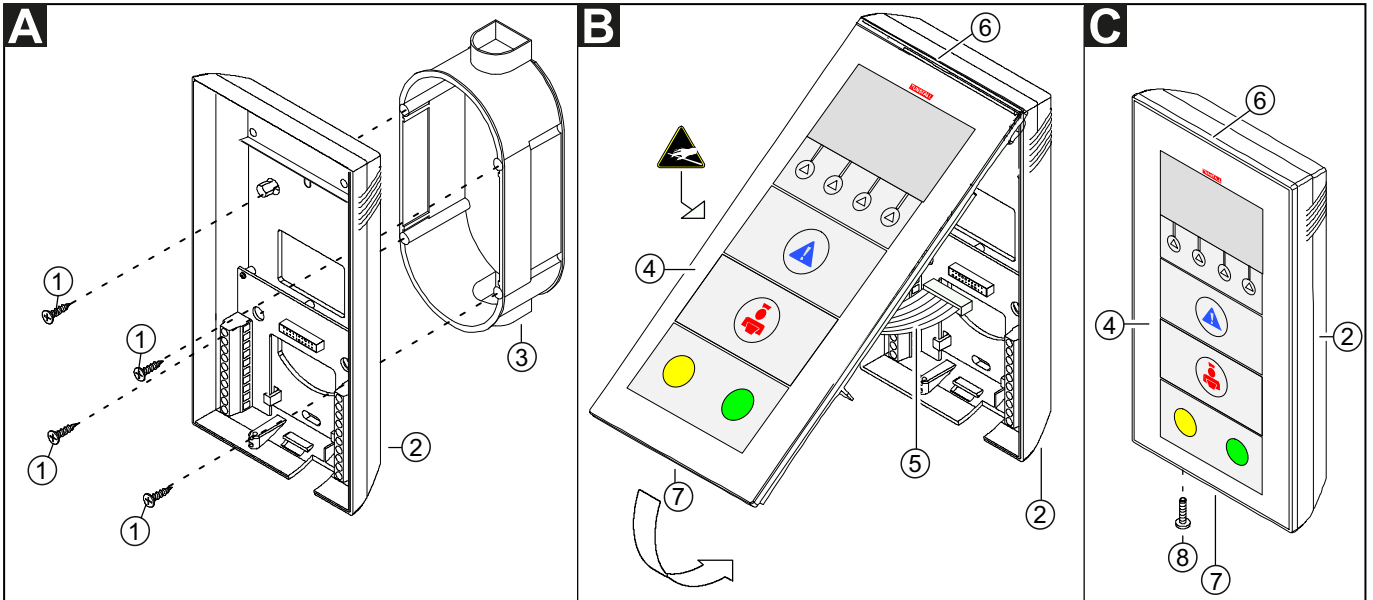Next, connect the connection cables to the connection field in the pattress [2] according to page "Connections".

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- 1 - * Four back box screws
- 2 - Pattress
- 3 - * Back box
- 4 - Front panel

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* Not included with RoomTerminal delivery

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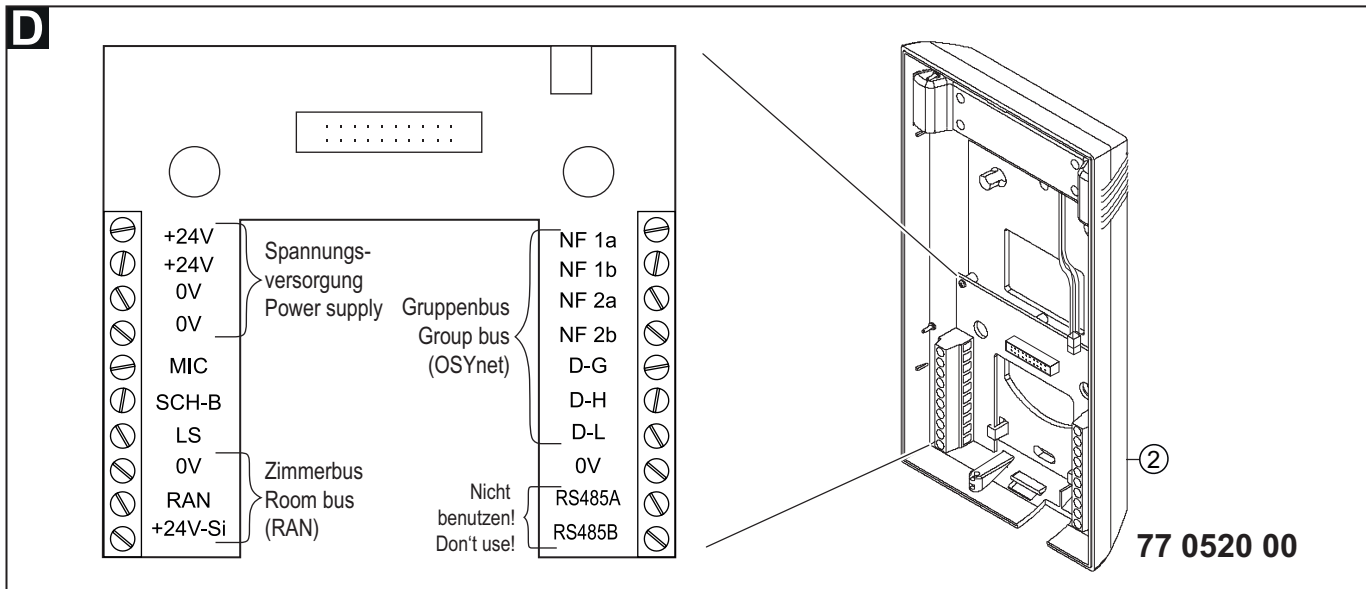
D Anschlüsse**B Mounting the front panel**

1. Plug the flat cable [5] of the front panel [4] into the appropriate socket on the connection field in the pattress [2].
2. Place front panel [4] onto the top of the pattress [2] creating a hinge [6].
3. Press the front panel downwards onto the pattress with low pressing power until the catch [7] engages. Make sure that the flat cable [5] is not pinched.
4. For improved security the security screw [8] can be fitted into the hole below the yellow presence button.

Finally, the RoomTerminal has to be configured according to the pages "Configuration Instructions".

C Dismantling

1. If a security screw [8] is fitted into the base of the RoomTerminal, remove it.
2. Use a screwdriver to push the catch [7] upwards until the front panel [4] loosens.
3. Remove the front panel [4].
4. Disconnect the flat cable [5] from the pattress [2].
5. Lay down the front panel [4] protected.
Caution: Electrostatic sensitive components!

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
Connection capacity	0.20 – 2.50 mm ²
Screwdriver blade	0.6 x 3.5 mm
Insulation strip length	6 mm

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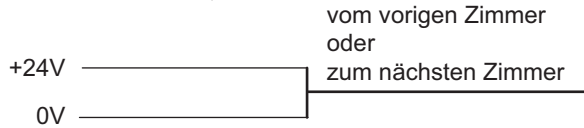
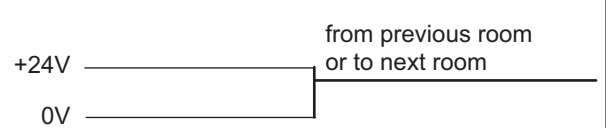
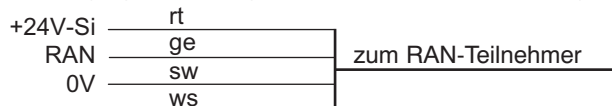
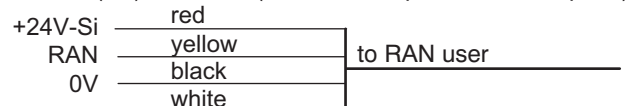
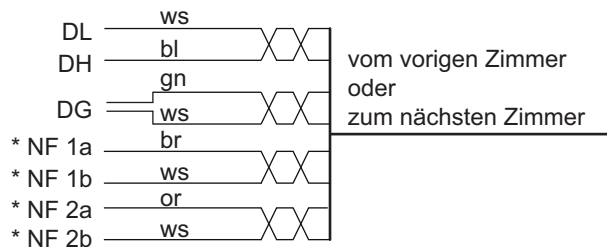
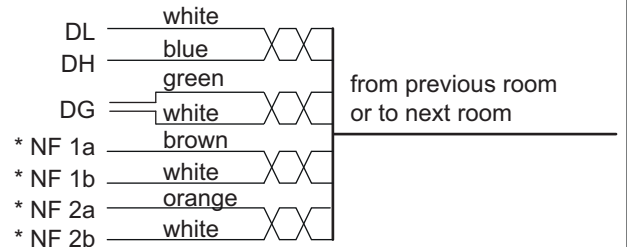
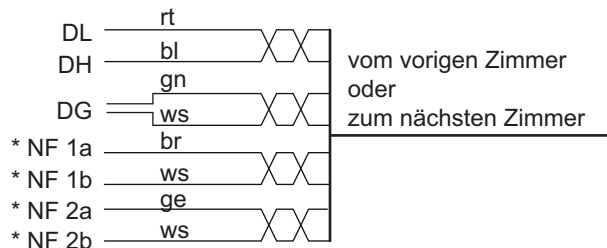
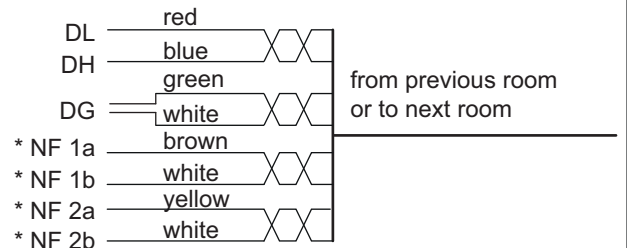
* Note! The NF connections are not required in the following installations: There are no ComTerminals (= terminals with speech) connected to this group bus OSYnet AND there shall not be connected any ComTerminals to this group bus OSYnet in the future.

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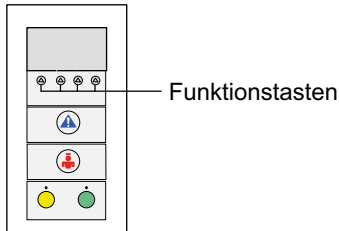
1. Start configuration menu

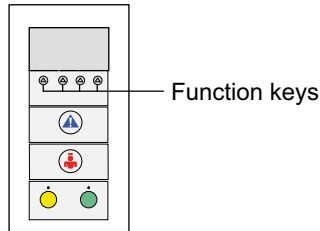- Press the left and right function keys simultaneously (3 sec.) until the configuration menu is shown in the display.

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Symbols in the configuration menu

●	Circle key	Set the marked menu item.
▲	Upwards arrow key	Scroll upwards through the list.
▼	Downwards arrow key	Scroll downwards through the list.
⌂	Home key	Cancel, without storing.

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Menu items

- Language: Selecting the user language, e.g. English.
- Room type: Selecting the room type, e.g. patient room.
- RAN > RAN number: Setting of number of room devices (= RAN number).
- RAN > Test RAN: Test if room devices are ready to operate and if they are correctly connected to the RAN.
- RAN > Status: Check whether the stored RAN number is equal to the number of operational room devices.
- Address: Setting of room address.
- Contrast: Function use only for Tunstall technicians.
- Key sound: Switching the key sound of the RoomTerminal on or off. "Key sound On" (factory setting) is recommended.
- Info: Displaying the RoomTerminal's software revision.
- Reset: Function use only for Tunstall technicians. (Restart the RoomTerminal).

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Selecting the room type

1. Mark "Room type" using the arrow keys. Then press the circle key.
2. Mark the desired room type using the arrow keys:
 - ♦ Patient room
 - ♦ Patient room with cancel key for WC call at RoomTerminal
 - ♦ Staff room (emergency calls are signalled as normal calls)
 - ♦ Connection terminal (keys and display at RoomTerminal non-operational)
3. Press the circle key to set the selection.

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1. Mark “RAN” using the arrow keys. Then press the circle key.
2. Mark “RAN number” using the arrow keys. Then press the circle key.
3. Mark the desired RAN number using the arrow keys.
4. Press the circle key to set the selection.

Setting of room address (0 – 110)

1. Mark “Address” using the arrow keys. Then press the circle key.
2. Mark the desired room address using the arrow keys.
3. Press the circle key to set the selection.

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4. Checking the room bus RAN**Status**

Check whether the set RAN number (refer to “Setting of RAN number”) equals to the number of operational room devices.

1. Mark “RAN” using the arrow keys. Then press the circle key.
2. Mark “Status” using the arrow keys. Then press the circle key.

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3. Press the home key to end. In case of an **error** perform a “Test RAN”.

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3. Compare the number of flashing room devices with the set RAN number. If the set RAN number is wrong, correct the RAN number setting after the RAN test as described in section “Setting of RAN number”.
4. End of test: Press the home key.

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- 1 - * Four back box screws
2 - Pattress
3 - * Back box
4 - Front panel
5 - Flat cable

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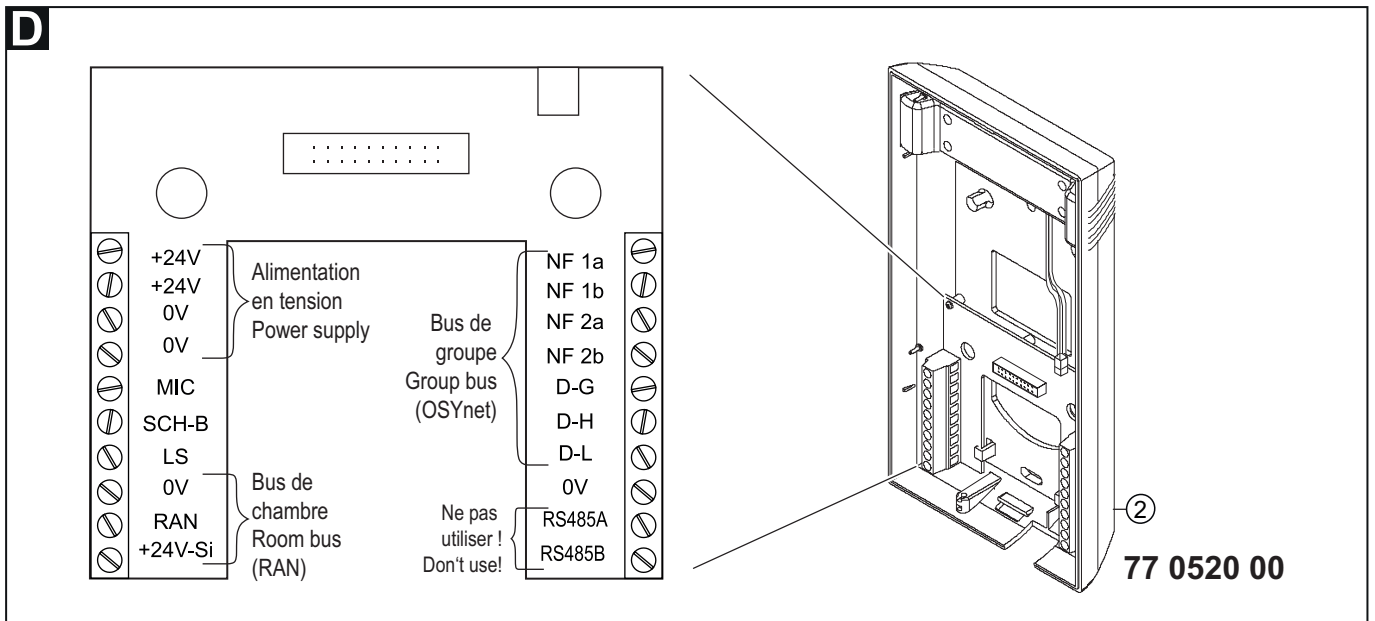
D Raccordements**B Mounting the front panel**

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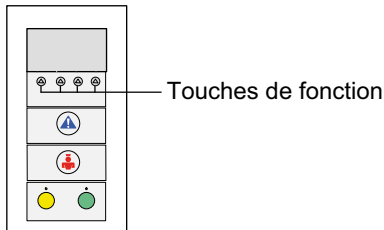
Symbols in the configuration menu

●	Circle key	Set the marked menu item.
▲	Upwards arrow key	Scroll upwards through the list.
▼	Downwards arrow key	Scroll downwards through the list.
🏠	Home key	Cancel, without storing.

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Menu items

- Language: Selecting the user language, e.g. English.
 Room type: Selecting the room type, e.g. patient room.
 RAN > RAN number: Setting of number of room devices (= RAN number).
 RAN > Test RAN: Test if room devices are ready to operate and if they are correctly connected to the RAN.
 RAN > Status: Check whether the stored RAN number is equal to the number of operational room devices.
 Address: Setting of room address.
 Contrast: Function use only for Tunstall technicians.
 Key sound: Switching the key sound of the RoomTerminal on or off. "Key sound On" (factory setting) is recommended.
 Info: Displaying the RoomTerminal's software revision.
 Reset: Function use only for Tunstall technicians. (Restart the RoomTerminal).

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Selecting the room type

- Mark "Room type" using the arrow keys. Then press the circle key.
- Mark the desired room type using the arrow keys:
 - Patient room
 - Patient room with cancel key for WC call at RoomTerminal
 - Staff room (emergency calls are signalled as normal calls)
 - Connection terminal (keys and display at RoomTerminal non-operational)
- Press the circle key to set the selection.

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Setting of RAN number (0 – 30) (Room Area Network)

RAN number = Number of devices in the room (switches, connection sockets, room lamps, etc.) that are connected via RAN. (**No** pear push switch).

1. Mark "RAN" using the arrow keys. Then press the circle key.
2. Mark "RAN number" using the arrow keys. Then press the circle key.
3. Mark the desired RAN number using the arrow keys.
4. Press the circle key to set the selection.

Setting of room address (0 – 110)

1. Mark "Address" using the arrow keys. Then press the circle key.
2. Mark the desired room address using the arrow keys.
3. Press the circle key to set the selection.

3. Entering of optional settings

The remaining menu items provide optional settings. Set these if required.

4. Checking the room bus RAN**Status**

Check whether the set RAN number (refer to section "Setting of RAN number") equals to the number of operational room devices.

1. Mark "RAN" using the arrow keys. Then press the circle key.
2. Mark "Status" using the arrow keys. Then press the circle key.

Display: **OK**: The set RAN number is equal to the number of operational room devices.

Display: **Error**: The set RAN number is not equal to the number of operational room devices.

3. Press the home key to end. In case of an **error** perform a "Test RAN".

Test RAN

Test if room devices are ready to operate and if they are correctly connected to the RAN.

Check the correct setting of the RAN number.

1. Mark "RAN" using the arrow keys. Then press the circle key.
2. Test start: Mark "Test RAN" using the arrow keys. Then press the circle key.

The LEDs of all connected room devices must flash. Non-flashing devices are wrongly connected or defective. Replace defective devices. Correct any wrong connection.

3. Compare the number of flashing room devices with the set RAN number. If the set RAN number is wrong, correct the RAN number setting after the RAN test as described in section "Setting of RAN number".
4. End of test: Press the home key.

5. Exit the configuration menu

When all settings are made and all tests are completed, you have to exit the configuration menu:

- Press home key several times until normal operation display appears.

NOTE! If for a period of three minutes no function key is pressed, the system will leave the configuration menu.