Telefonanschaltrelais, Best.-Nr. 11 5350 00

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Telephone interface relay, order no. 11 5350 00

for connecting analogues telephones to a nurse call system. A telephone call will be displayed as a call in the nurse call system.

- Call voltage: 32 - 80 V AC
- Power supply: 5 - 40 V
- ON/OFF switch (ON = EIN, OFF = AUS)
- Output: N/C, N/O contact
- Bridging of call intervals can be set
- Power-on time can be set.

Dimensions (HxWxD): 70 x 35 x 70 mm

Mounting on a 35 mm top hat rail.

Note! The complete installation of the system is described in the technical manual.

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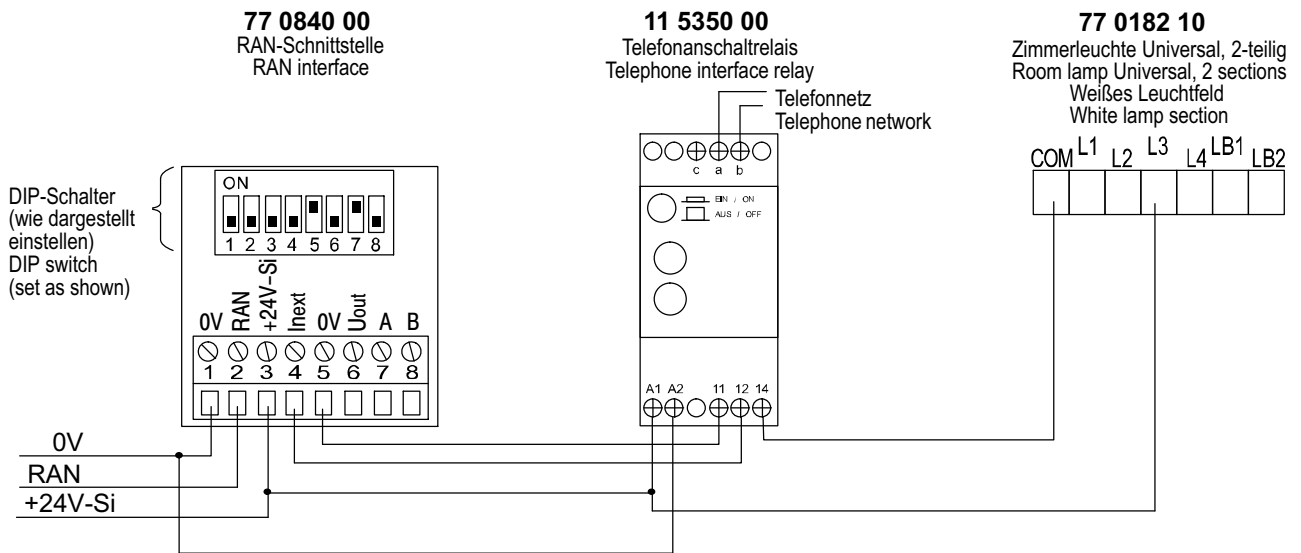
A Mounting

1. Power down the equipment.
2. Place the telephone interface relay on the top hat rail and click it into place.
3. Connect as shown in the diagram below.
4. Restart the equipment.
5. Switch on the telephone interface relay by pressing down the ON/OFF switch.

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When the telephone interface relay is operational, the switching contact is closed as soon as the telephone signal voltage is applied. It depends on the setting of the two potentiometers "bridging of call intervals" and "power-on time" how long the switching contact remains closed.

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The potentiometer for the bridging of call intervals sets the period the switching contact remains closed longer than the telephone interface relay is actuated. The setting range is linear and lies between 0 and 12 seconds. If the selected time is longer than the actuation intervals by the telephone call voltage the switching contact remains closed without interruption until the actuation stops (end of call). If the selected time is shorter than the actuation interval the switching time of the contact results from the signal period plus the selected bridging time.

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The potentiometer for power-on time setting allows to limit the time the switching contact remains closed. The setting range is not linear and lies between 0.25 and about 12 seconds. If the potentiometer is set for permanent ("Dauer") the switching contact is not limited and remains closed up to the end of actuation.

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© Tunstall GmbH, Okoliten 66, D-48291 Telgte, www.tunstall.de