Steckvorrichtung ComStation, Best.-Nr. 77 0452 30 A

Steckvorrichtung ComStation, Best.-Nr. 77 0452 30 F

Steckvorrichtung ComStation, Best.-Nr. 74 0452 30

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Connection socket ComStation, order no. 77 0452 30 A

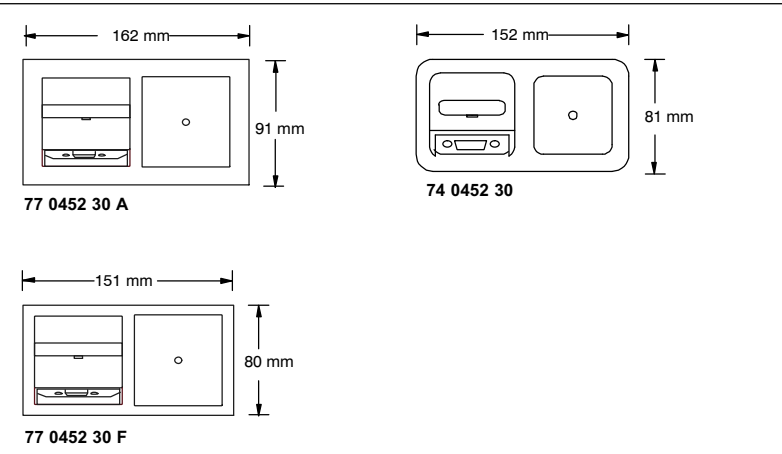
Connection socket ComStation, order no. 77 0452 30 F

Connection socket ComStation, order no. 74 0452 30


The connection sockets differ only in their external design. All connection sockets can be used for the connection of the devices mentioned in these installation instructions. Observe the different connection diagrams for the different applications.

- Preferably made for wall mounting.
- Suitable for 2-gang back box (back box not included with delivery)
- 15 pole socket as data socket
- Prepared for 15 pole plug with mechanical locking mechanism
- 4 plug-in terminals up to 2.5 mm² for connection of the 24 V supply voltage
- Connection as plug-in terminals for solid wire of ø 0.55 - 1 mm.
- Contact current: max. 3 A


Note! The complete installation of the system is described in the technical manual.

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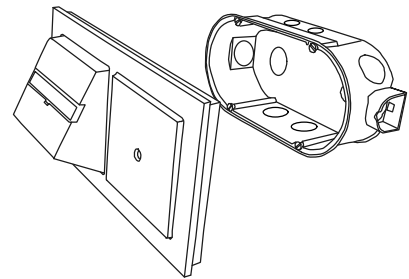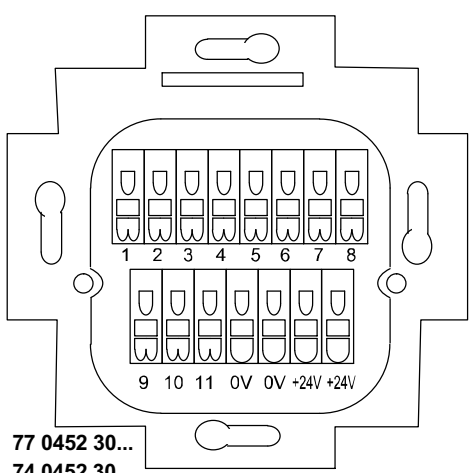
* push-wire connector, 5 pole (00 0210 21)
 wire cross section: 0.5 - 2.5 mm²

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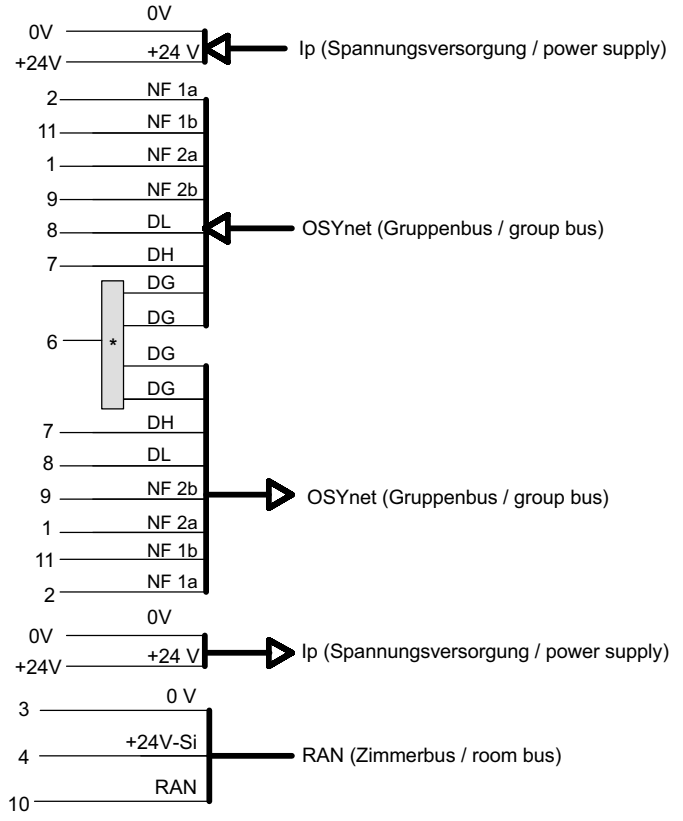
Note for ComStation^T (77 0606 20) and RoomTerminal Flamenco, desktop installation (77 0521 00)! The connections NF 1a, NF 1b, NF 2a and NF 2b are not required in the following installations: There shall neither be connected a ComStation^{CT} (with speech, 77 0606 00) nor a ComTerminal Flamenco, desktop installation (77 0511 00) to this connection socket in the future.

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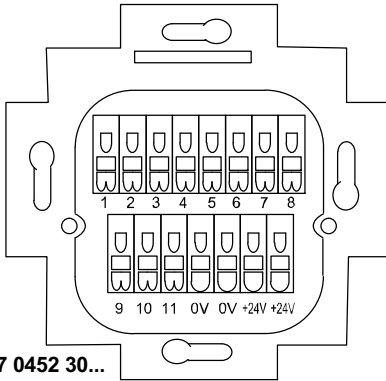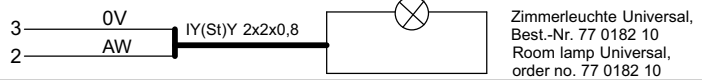
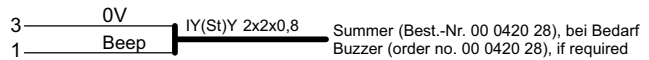
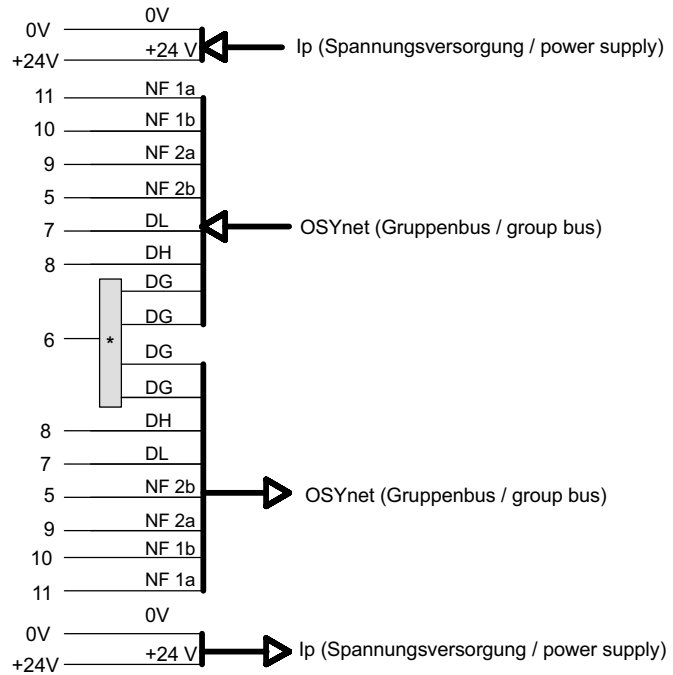
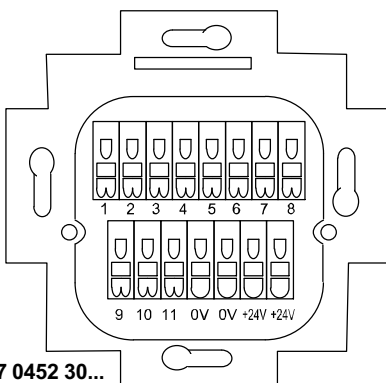
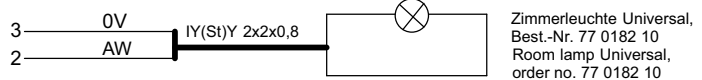
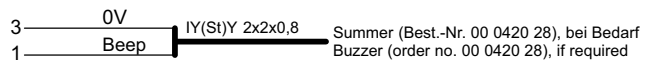
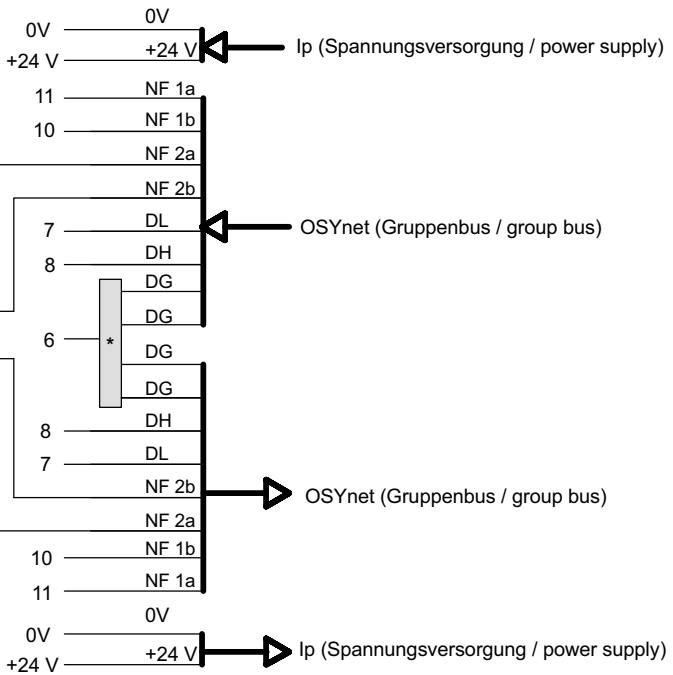
Connection socket ComStation, order no. 77 0452 30 A

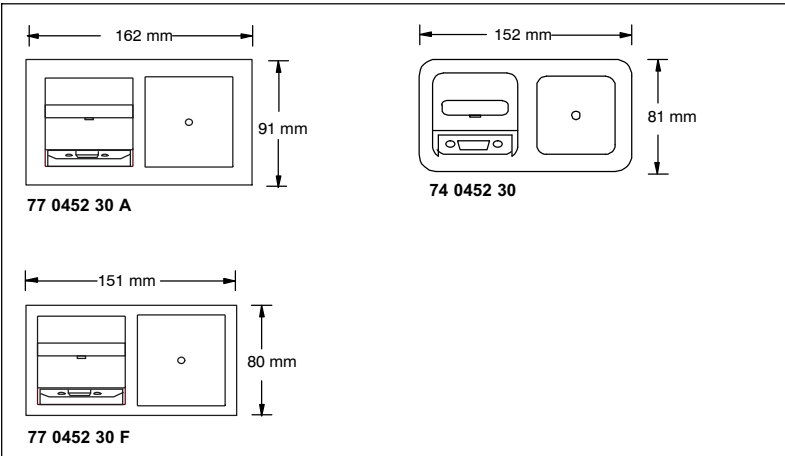
Connection socket ComStation, order no. 77 0452 30 F

Connection socket ComStation, order no. 74 0452 30


The connection sockets differ only in their external design. All connection sockets can be used for the connection of the devices mentioned in these installation instructions. Observe the different connection diagrams for the different applications.

- Preferably made for wall mounting.
- Suitable for 2-gang back box (back box not included with delivery)
- 15 pole socket as data socket
- Prepared for 15 pole plug with mechanical locking mechanism
- 4 plug-in terminals up to 2.5 mm² for connection of the 24 V supply voltage
- Connection as plug-in terminals for solid wire of ø 0.55 - 1 mm.
- Contact current: max. 3 A

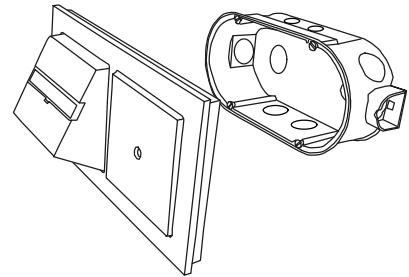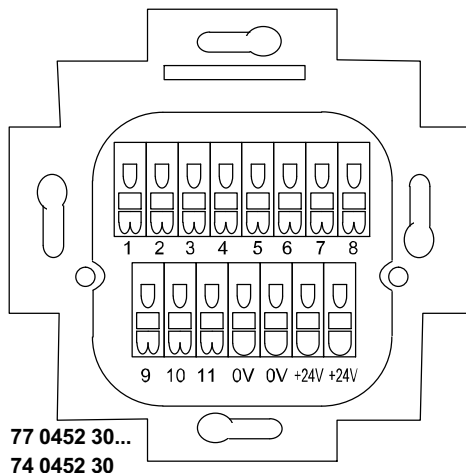
 **Note!** The complete installation of the system is described in the technical manual.

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© Tunstall GmbH, Okoliten 66, D-48291 Telgte, www.tunstall.de