## DE - Installationsanleitung

### Zimmerleuchte, 3-teilig, Best.-Nr. 77 0170 00

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## EN - Installation Instructions

### Room lamp, 3 sections, order no. 77 0170 00

Signalling of all call types and staff presences.

### Room lamp, 3 sections, glass decor order no. 77 0175 00

Same as 77 0170 00, but decorative glass frame.

### Room lamp cardiac alarm, WC, order no. 77 0170 01

Complement to room lamp 77 0170 00 for explicit signalling of cardiac alarm and/or WC calls. Mounting either in the corridor next to 77 0170 00 or next to the WC door in the patient room.

### Room lamp cardiac alarm, WC, glass decor, 77 0175 01

Same as 77 0170 01, but as complement to 77 0175 00, decorative glass frame.

### Room lamp, 4 sections, order no. 77 0170 10

Signalling of all call types and staff presences as well as additional display for WC call.

### Room lamp, 4 sections, glass decor, order no. 77 0175 10

Same as 77 0170 10, but decorative glass frame.

**NOTE!** The complete installation of the system is described in the Technical Manual.

**CAUTION!** The printed circuit board and the LED modules include electrostatic sensitive components. Avoid touching.

## A Connection

1. Strip the connecting cable in the back box to a suitable length.
  2. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
  3. Connect the wires to the connector [3] according to fig. **A**.
- CAUTION!** The unused wires must not be connected through.

## B DIP switch, 4 pole

### DIP switch 1 – 3: Channel coding

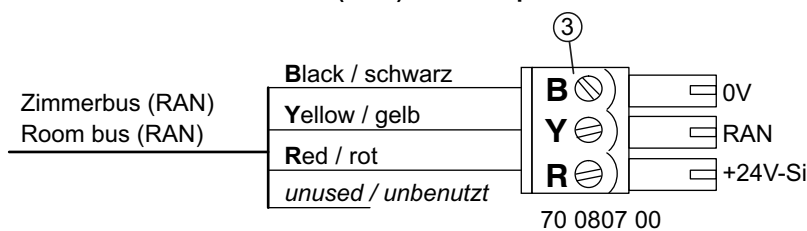
If functional units are requested, the room lamps have to be assigned to different channels. For the channel number to be set please refer to the installation plan. Set the channel number according to fig. **B**.

### DIP switch 4: Signalling

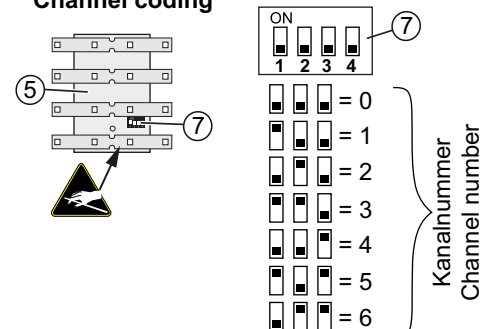
OFF = Fresh calls of the call category “Calls” are signalled with a short attention blink. This makes it possible to distinguish between fresh calls and answered calls. (OFF = factory setting)

ON = Fresh calls of call category “Calls” are signalled as continuous light according to the German standard DIN VDE 0834-1.

## A Anschluss an Zimmerbus (RAN) ohne Sprechen Connection to room bus (RAN) without speech

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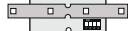
Lamp:	77 0170 00 77 0175 00	77 0170 01 77 0175 01	77 0170 10 77 0175 10
	red: calls	blue: cardiac alarm	red: calls
	green: staff 1	blue: cardiac alarm	green: staff 1
	yellow: staff 2	white: WC call	yellow: staff 2
	no LED module	no LED module	white: WC call

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At the point of delivery the room lamp is assembled and must be dismantled as follows, see fig. D:

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Wall installation on a one-gang back box [1] fixed with screws on the lateral holes of the back box \*):

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1. Prepare the wires and connect them to the connector [3] (see section "Connection").
2. Plug the connector [3] from the rear onto the PCB [5] (in the rear of the housing [2]).
3. Bolt the housing [2] to the back box [1] with the screws [4]. The arrow on the rear of the housing points toward the top.
4. Press the light dome with insert module [8] onto the decorative frame until it latches in audibly.

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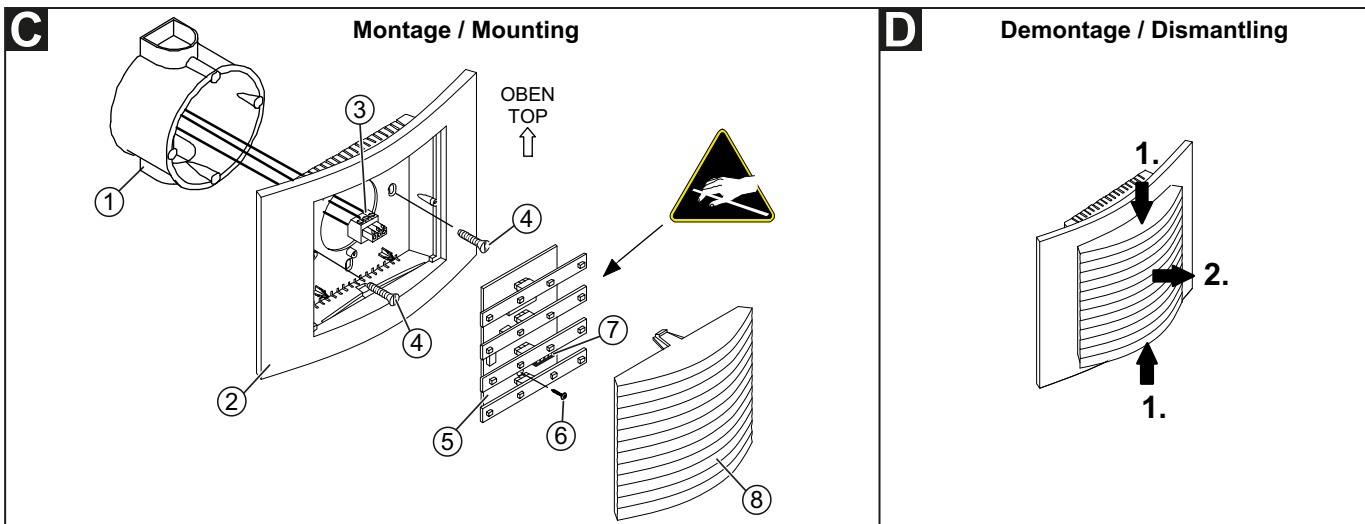
\*) **NOTE!** To mount the room lamp on the vertical holes of the back box, the PCB must be removed temporarily:

1. Loosen the fastening screw [6] of the PCB and remove the PCB [5] (first bottom, then top).
2. The PCB [5] is installed in the reverse order.

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- [1] \* Back box
- [2] Housing (glass decorative frame looks different)
- [3] \* Connector (70 0807 00)
- [4] \* Back box screws
- [5] Printed circuit board (PCB) with LED modules

- [6] Fastening screw for the PCB
  - [7] DIP switch
  - [8] Light dome with insert module
- \* Not included in the scope of delivery, please order separately.

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## A Connection

- Strip the connecting cable in the back box to a suitable length.
  - Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
  - Connect the wires to the connector [3] according to fig. A.
- CAUTION!** The unused wires must not be connected through.

## B DIP switch, 4 pole

### DIP switch 1 – 3: Channel coding

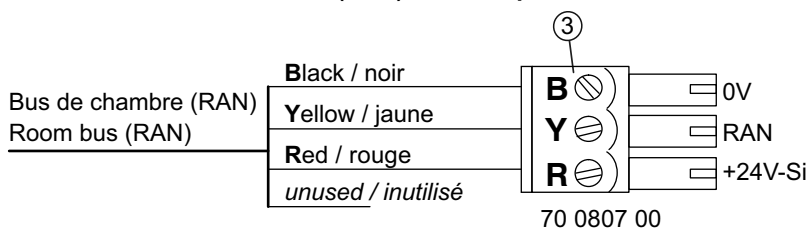
If functional units are requested, the room lamps have to be assigned to different channels. For the channel number to be set please refer to the installation plan. Set the channel number according to fig. B.

### DIP switch 4: Signalling

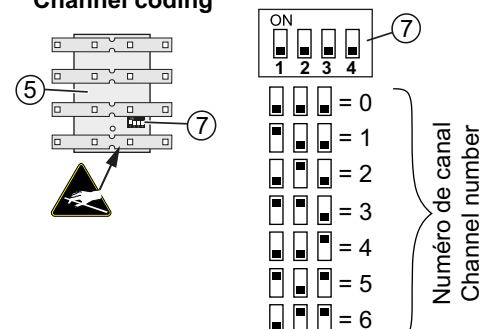
OFF = Fresh calls of the call category "Calls" are signalled with a short attention blink. This makes it possible to distinguish between fresh calls and answered calls. (OFF = factory setting)

ON = Fresh calls of call category "Calls" are signalled as continuous light according to the German standard DIN VDE 0834-1.

## A Raccordement au bus de chambre (RAN) sans phonie Connection to room bus (RAN) without speech

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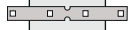
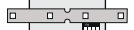
### Mounting

At the point of delivery the room lamp is assembled and must be dismantled as follows, see fig. D :

- Lightly compress the light dome [8] from top and bottom. Then pull off the light dome [8].

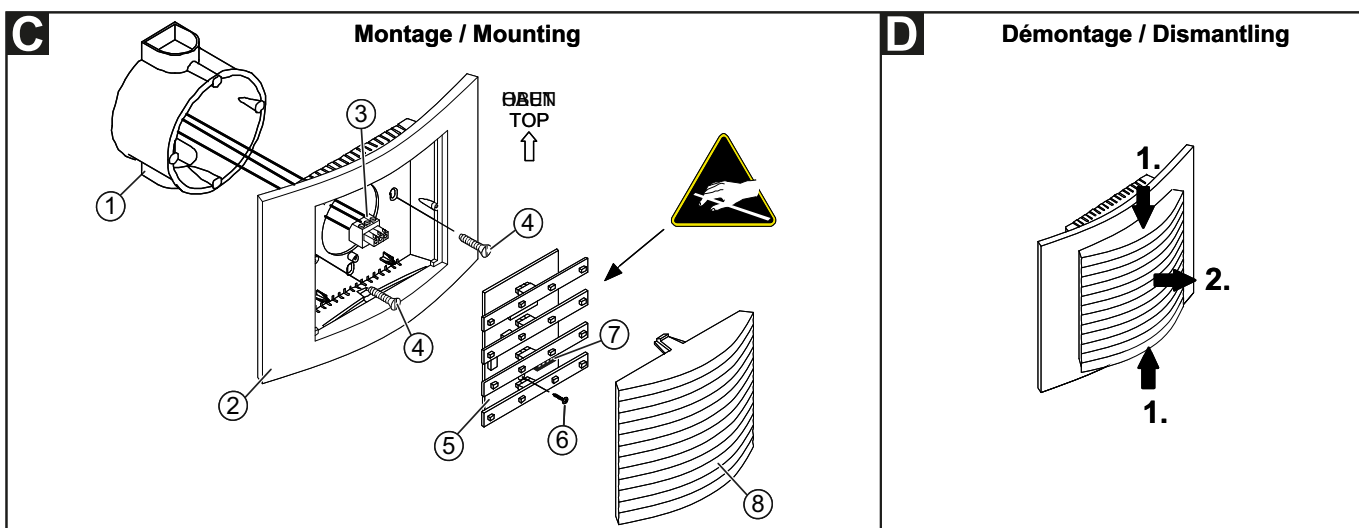
Wall installation on a one-gang back box [1] fixed with screws on the lateral holes of the back box \*) :

- Prepare the wires and connect them to the connector [3] (see section "Connection").
- Plug the connector [3] from the rear onto the PCB [5] (in the rear of the housing [2]).
- Bolt the housing [2] to the back box [1] with the screws [4]. The arrow on the rear of the housing points toward the top.
- Press the light dome with insert module [8] onto the decorative frame until it latches in audibly.

**\*) NOTE:** To mount the room lamp on the vertical holes of the back box, the PCB must be removed temporarily:

- Loosen the fastening screw [6] of the PCB and remove the PCB [5] (first bottom, then top).
- The PCB [5] is installed in the reverse order.

**NOTE!** The room lamp, 4 sections, equals to 1 RAN user to be set at the room terminal. The previous model equalled to 2 RAN users. Distinctive mark of the previous model: It has no DIP switch [7].

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- [1] \*Back box  
[2] -Housing (glass decorative frame looks different)  
[3] \* Connector (70 0807 00)  
[4] \*Back box screws  
[5] Printed circuit board (PCB) with LED modules

- [6] Fastening screw for the PCB  
[7] DIP switch  
[8] Light dome with insert module  
\* Not included in the scope of delivery, please order separately.