## DE - Installationsanleitung

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## EN - Installation Instructions

### Room lamp, 3 sections, with doorplate, order no. 77 0171 00

Signalling of all call types and staff presences. Doorplate as label field for room designation.

### Room lamp, 4 sections, with doorplate, order no. 77 0171 10

Same as 77 0171 00, but with additional display for WC call.

**NOTE!** The complete installation of the system is described in the Technical Manual.

**CAUTION!** The printed circuit board and the LED modules include electrostatic sensitive components. Avoid touching.

#### A Connection

1. Strip the connecting cable in the back box to a suitable length.
2. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
3. Connect the wires to the connector [3] according to fig. **A**.

**CAUTION!** The unused wires must not be connected through.

#### B DIP switch, 4 pole

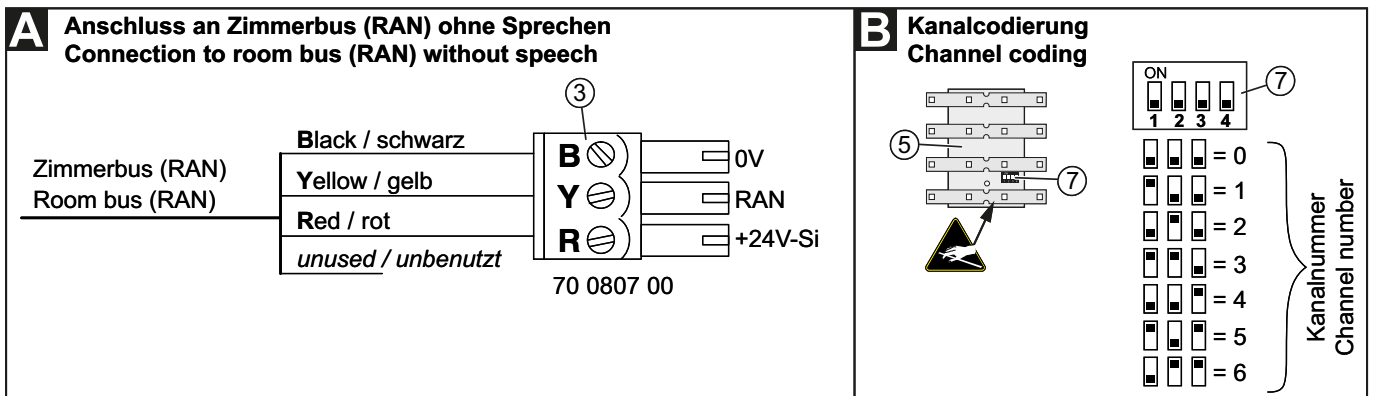
##### DIP switch 1 – 3: Channel coding

If functional units are requested, the room lamps have to be assigned to different channels. For the channel number to be set please refer to the installation plan. Set the channel number according to fig. **B**.

##### DIP switch 4: Signalling

OFF = Fresh calls of the call category “Calls” are signalled with a short attention blink. This makes it possible to distinguish between fresh calls and answered calls. (OFF = factory setting)

ON = Fresh calls of call category “Calls” are signalled as continuous light according to the German standard DIN VDE 0834-1.

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#### LED module colours

Lamp:	77 0171 00	77 0171 10
	red: calls	red: calls
	green: staff 1	green: staff 1
	yellow: staff 2	yellow: staff 2
	no LED module	white: WC call

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**C Mounting**

At the point of delivery the room lamp is assembled and must be dismantled as follows, see fig. D:

- Lightly compress the light dome [8] from top and bottom. Then pull off the light dome [8].

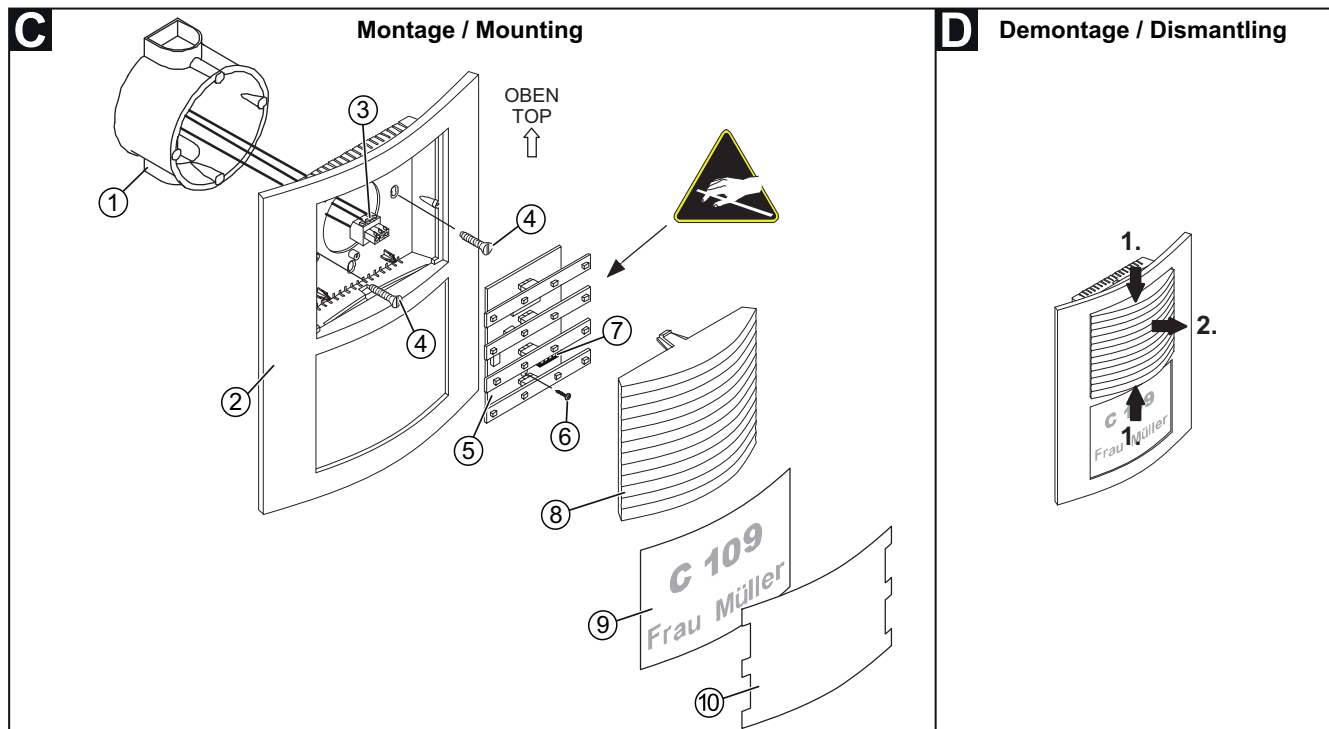
Wall installation on a one-gang back box [1] fixed with screws on the lateral holes of the back box \*):

1. Prepare the wires and connect them to the connector [3] (see section "Connection").
2. Plug the connector [3] from the rear onto the PCB [5] (in the rear of the housing [2]).
3. Bolt the housing [2] to the back box [1] with the screws [4]. The arrow on the rear of the housing points toward the top.
4. Press the light dome with insert module [8] onto the decorative frame until it latches in audibly.
5. Insert the label strip [9] and the protection cover [10].

\*) **NOTE!** To mount the room lamp on the vertical holes of the back box, the PCB must be removed temporarily:

1. Loosen the fastening screw [6] of the PCB and remove the PCB [5] (first bottom, then top).
2. The PCB [5] is installed in the reverse order.

**NOTE!** The room lamp, 4 sections, equals to 1 RAN user to be set at the room terminal. The previous model equalled to 2 RAN users. Distinctive mark of the previous model: It has no DIP switch [7].

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- [1] \*Back box  
[2] Housing  
[3] \*Connector (70 0807 00)  
[4] \*Back box screws  
[5] Printed circuit board (PCB) with LED modules  
[6] Fastening screw for the PCB

- [7] DIP switch  
[8] Light dome with insert module  
[9] \*Label strip (Height x Width: 70 x 92 mm)  
[10] Protection cover

\* Not included in the scope of delivery, please order separately.

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### A Connection

- Strip the connecting cable in the back box to a suitable length.
  - Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
  - Connect the wires to the connector [3] according to fig. A.
- CAUTION!** The unused wires must not be connected through.

### B DIP switch, 4 pole

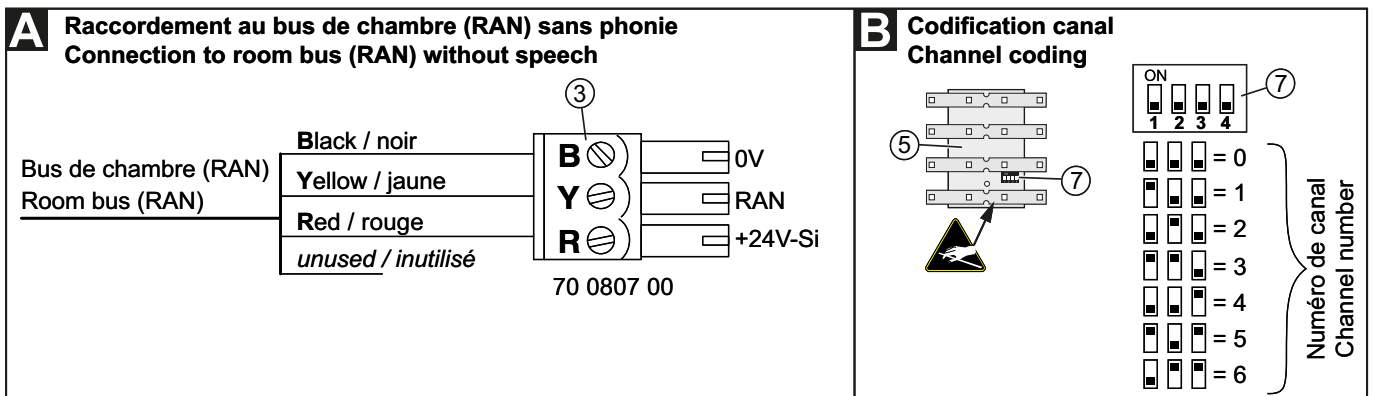
#### DIP switch 1 – 3: Channel coding

If functional units are requested, the room lamps have to be assigned to different channels. For the channel number to be set please refer to the installation plan. Set the channel number according to fig. B.

#### DIP switch 4: Signalling

OFF = Fresh calls of the call category "Calls" are signalled with a short attention blink. This makes it possible to distinguish between fresh calls and answered calls. (OFF = factory setting)

ON = Fresh calls of call category "Calls" are signalled as continuous light according to the German standard DIN VDE 0834-1.

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**C Mounting**

At the point of delivery the room lamp is assembled and must be dismantled as follows, see fig. D :

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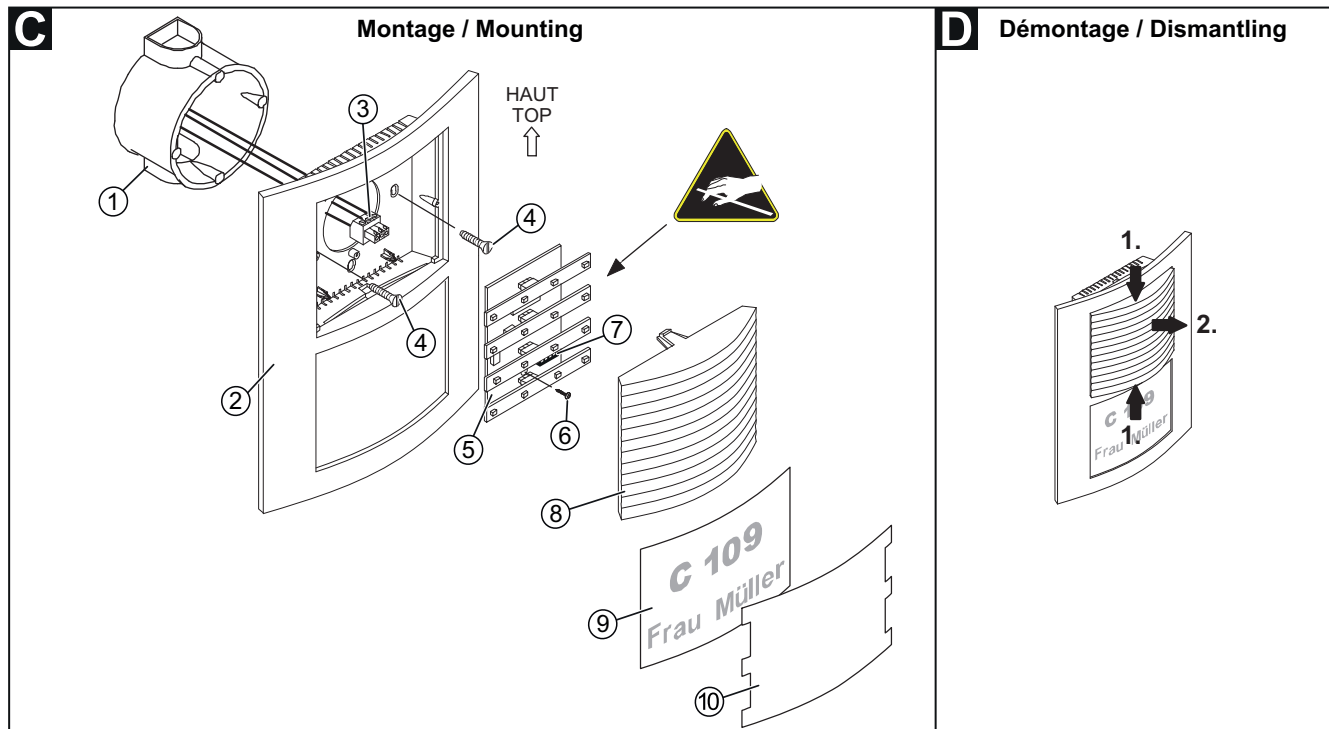
Wall installation on a one-gang back box [1] fixed with screws on the lateral holes of the back box \*) :

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**\*) NOTE:** To mount the room lamp on the vertical holes of the back box, the PCB must be removed temporarily:

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- [1] \*Back box  
 [2] Housing  
 [3] \*Connector (70 0807 00)  
 [4] \*Back box screws  
 [5] Printed circuit board (PCB) with LED modules  
 [6] Fastening screw for the PCB

- [7] DIP switch  
 [8] Light dome with insert module  
 [9] \*Label strip (Height x Width: 70 x 92 mm)  
 [10] Protection cover

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