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Connection socket with call switch, order no. 70 0171 60...*)

...*) The letter at the end of the order number represents the frame of the switch:

A = Frame dimensions (HxW): 91 x 91 mm

F = Frame dimensions (HxW): 80 x 80 mm

C = Frame dimensions (HxW): 107 x 107 mm.

CAUTION! Frame C is made from real glass!

Intended use

Connection socket with two 8-pole sockets for connection of call devices like pear push switches or medical monitoring devices. The connection of an additional external call device incl. location light and reassurance light is made on the back side with a special connection cable. Two outputs for light control.

Pressing the call button, triggering a connected pear push switch or triggering the call device connected to the external call input triggers the "Call" call type. An emergency call is triggered when the presence is activated. A medical monitoring device connected to a socket via a diagnostic connection cable (70 0812 10) triggers a diagnostic call.

Connection socket with call switch, WC order no. 70 0171 62...

As 70 0171 60..., except for the following differences:

Pressing the call button, triggering a connected pear push switch or triggering the call device connected to the external call input triggers the "WC call" call type. A WC emergency call is triggered when the presence is activated.

Connecting a medical monitoring device to a socket via a diagnostic connection cable (70 0812 10) is not permitted. The connection socket is not able to trigger a diagnostic call.

Connection socket with call switch, Alarm order no. 70 0171 65...

As 70 0171 60..., except for the following differences:

A call device connected to a socket via a diagnostic connection cable (70 0812 10) triggers the call type "Cardiac alarm". For information on how to connect the connection cable to the call device, please refer to the installation instructions supplied with the connection cable. Please note that this is an exceptional application of the connection cable and that the information on the diagnostic call in the installation instructions is therefore not applicable.

Connecting a medical monitoring device to a socket via a diagnostic connection cable (70 0812 10) is not permitted. The connection socket is not able to trigger a diagnostic call.

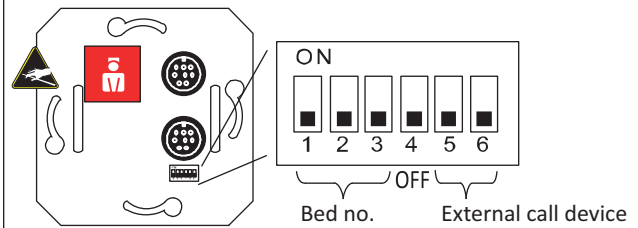
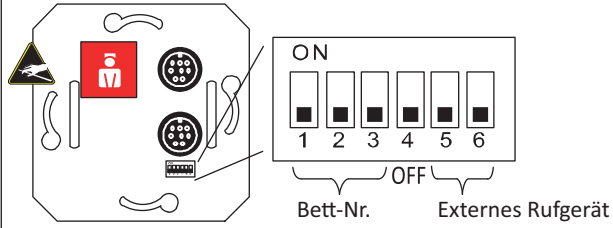
NOTE! The complete installation of the system is described in the Technical Manual.

CAUTION! The printed circuit board includes electrostatic sensitive components. Avoid touching.

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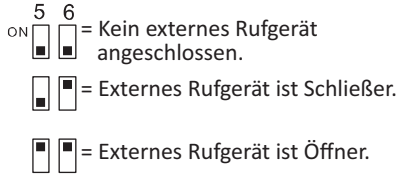
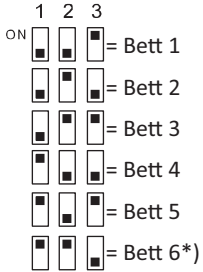
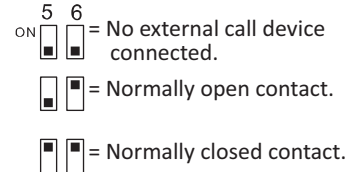
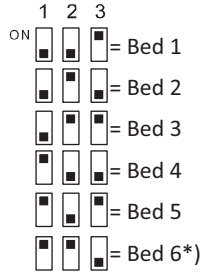
Mounting (→ fig. B)

1. Plug the 5-pole connector [2] into the 5-pole socket on the rear side of the PCB [4].
2. If an external call device shall be connected, plug the connector of the connection cable [3] for the external call device into the appropriate socket on the rear side of the PCB [4].
3. Screw the PCB [4] to the back box [1] with the back box screws [5].

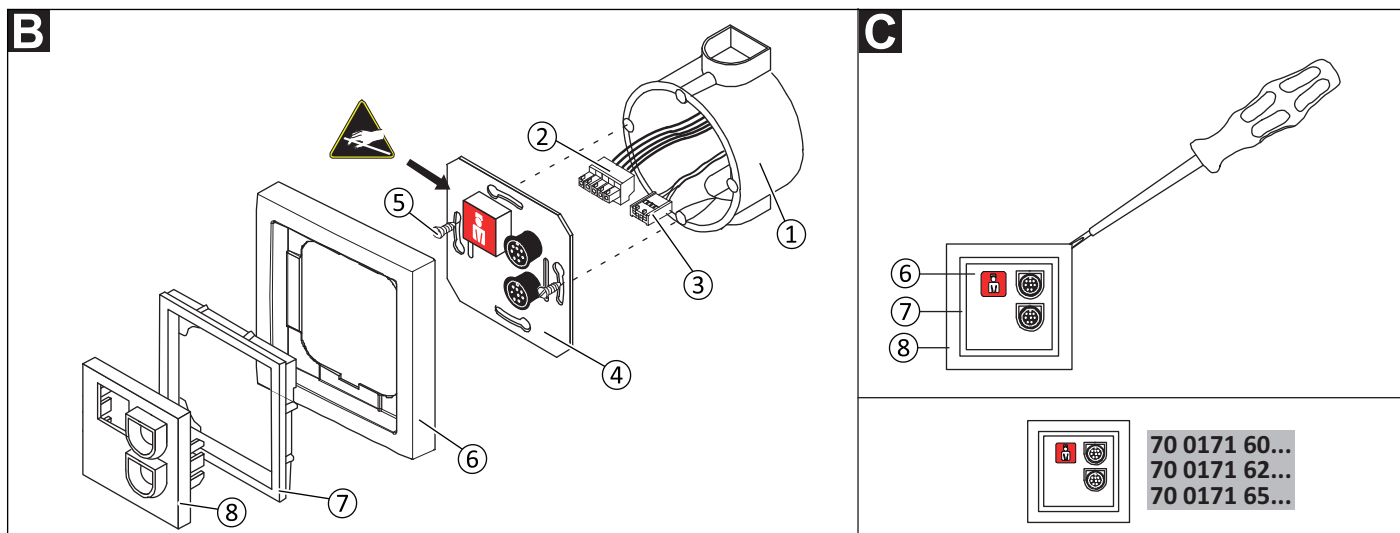
CAUTION! Risk of wedging or breaking the red call button! Proceed carefully when pressing the central plate [8] onto the PCB [4].

4. Press the central plate [8] together with the intermediate frame [7] and the frame [6] onto the PCB [4] until it engages audibly on both sides.

Dismantling (→ fig. C)

1. Lever frame [6] together with the intermediate frame [7] and the central plate [8] off the wall with a screw driver.
2. Undo the screws [5] and remove the PCB [4] from the back box [1].
3. Unplug the connectors from the rear of the PCB [4].

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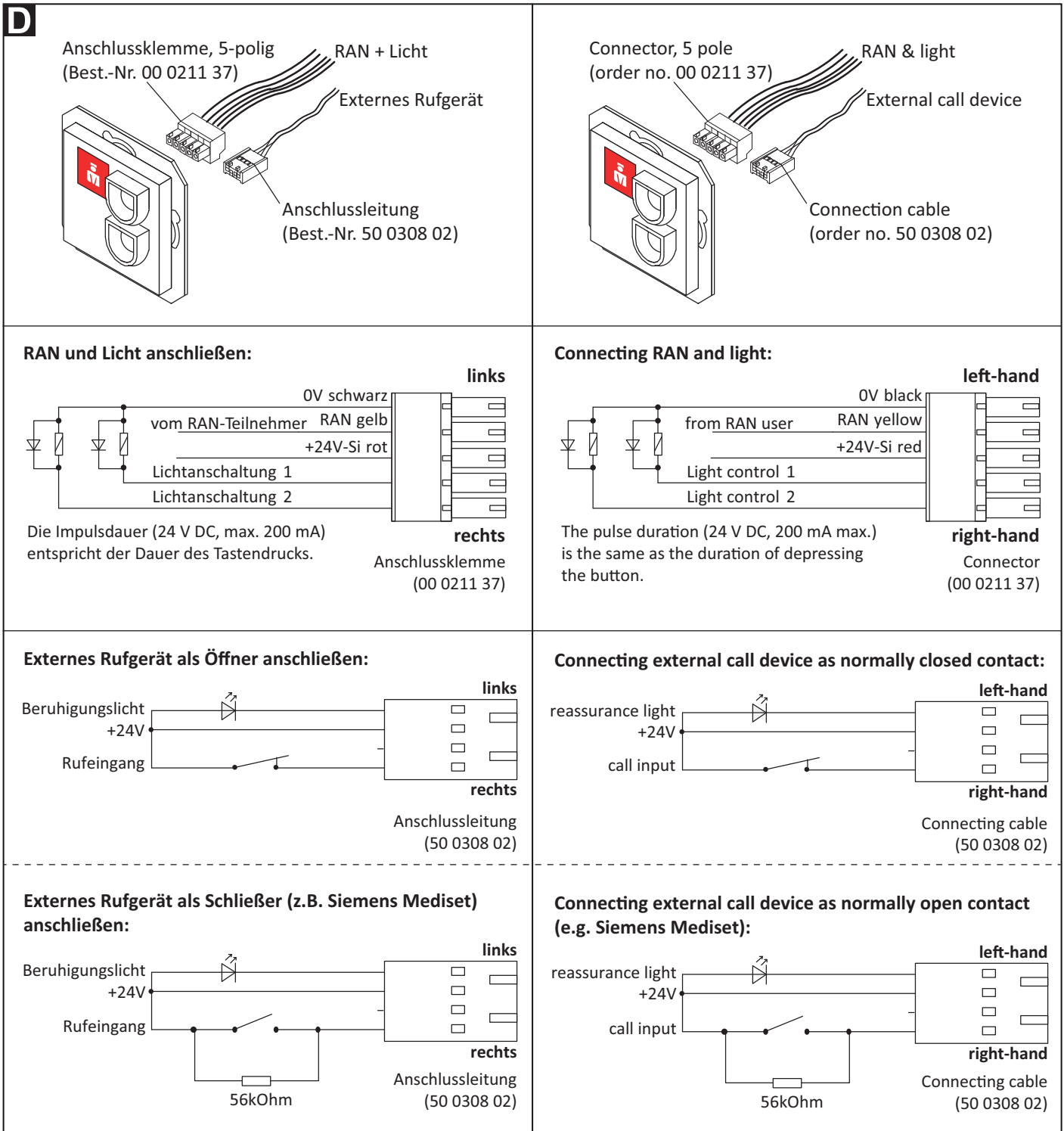
* Not included in the scope of delivery, please order separately.

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As external call device a normally-closed or a normally-open contact may be used. The external call device is connected via the special connection cable, order no. 50 0308 02 (length: 50 cm). The maximum cable length for the external call device is 2.5 m.

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Standby current consumption	21 mA
Central plate colour	Studio white, similar to RAL 9016
Central plate material	Polycarbonate

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© Tunstall GmbH, Orkotten 66, 48291 Telgte, Germany, www.tunstall.de