RAN-Schnittstelle

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RAN interface

The RAN interface is intended for connecting an external trigger device (normally closed or normally open contact) to the room bus (RAN) of a CONCENTO^{CARE} or CONCENTO^{PLUS} nurse call system. Thus the trigger device will trigger calls in the nurse call system. It is also possible to connect a LED of the trigger device to the RAN interface. The LED lights up as soon as the trigger device is triggered (reassurance light).

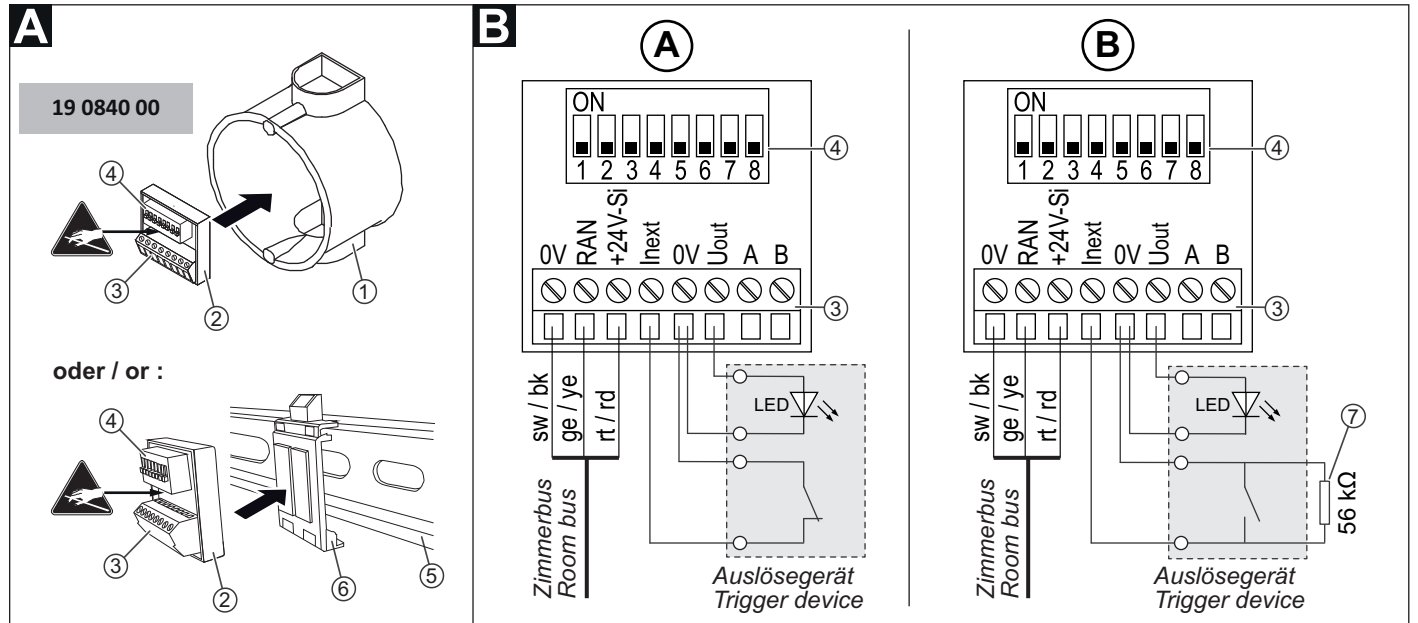
CONCENTO^{CARE} only: The triggering device can optionally be used as an activity trigger for the inactivity alarm function.

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NOTE! The complete installation of the system is described in the Technical Manual.

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CAUTION! The PCB is equipped with components that are at risk of being damaged electrostatically. Avoid direct contact.

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- [1] *Back box
- [2] RAN interface
- [3] Connector (screw-type terminal)
- [4] DIP switch, 8-pole

- [5] *Top hat rail, 35 mm
- [6] Top hat rail clip with two adhesive pads
- [7] Resistor 56 kΩ

* Not included in the scope of delivery, please order separately.

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A Mounting

Mounting in the vicinity of the trigger device (max. 5 m cable length), either in a back box or on a 35 mm top hat rail:

Back box installation

Simply insert the RAN interface [2] in the back box [1]. After the connection has been made, cover the back box with a blank plate or a line outlet.

Top hat rail installation

Stick the RAN interface [2] onto the adhesive pads on the top hat rail clip [6] and then snap both it and the top hat rail clip onto the top hat rail [5].

Dismantling

Back box installation: Remove the RAN interface [2] from the back box.

Top hat rail installation: Detach both the top hat rail clip [6] and the RAN interface [2] from the top hat rail [5].

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DIP-Schalter [4], Nummer 1 – 8 setzen

Auszulösende Rufart ***	DIP-Schalter Nr.					
	1	2	3	4	5	6
Raumruf **	OFF	OFF	OFF	OFF	OFF	OFF
Ruf Bett 1	ON	OFF	OFF	OFF	OFF	OFF
Ruf Bett 2	OFF	ON	OFF	OFF	OFF	OFF
Ruf Bett 3	ON	ON	OFF	OFF	OFF	OFF
Funkruf	OFF	OFF	OFF	ON	OFF	OFF
Passivalarm	OFF	OFF	OFF	OFF	ON	OFF
Serviceruf	OFF	OFF	OFF	ON	ON	OFF
Technischer Ruf	OFF	OFF	OFF	OFF	OFF	ON
Türalarm	OFF	OFF	OFF	ON	OFF	ON
Feueralarm	OFF	OFF	OFF	OFF	ON	ON

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EN - Installation Instructions

B Connection

- Strip the connection cables to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
 - **Normally closed contact:** Connect the wires according to connection diagram (A).
 - **Normally open contact:** Connect the wires according to connection diagram (B). Connect the 56 k Ω resistor according to connection diagram (B) (function: monitoring the contact).

Setting DIP switches [4], numbers 1 – 8

Call type to be triggered ***	DIP switch no.					
	1	2	3	4	5	6
Room call **	OFF	OFF	OFF	OFF	OFF	OFF
Call bed 1	ON	OFF	OFF	OFF	OFF	OFF
Call bed 2	OFF	ON	OFF	OFF	OFF	OFF
Call bed 3	ON	ON	OFF	OFF	OFF	OFF
Radio call	OFF	OFF	OFF	ON	OFF	OFF
Inactivity alarm	OFF	OFF	OFF	OFF	ON	OFF
Service call	OFF	OFF	OFF	ON	ON	OFF
Technical call	OFF	OFF	OFF	OFF	OFF	ON
Door alarm	OFF	OFF	OFF	ON	OFF	ON
Fire alarm	OFF	OFF	OFF	OFF	ON	ON

Trigger device contact type	DIP switch no.
Normally closed contact **	OFF
Normally open contact	ON

Cancelling the call that was triggered by the trigger device	DIP switch no.
Manual cancellation in the nurse call system. **	OFF
Automatic cancellation, if the trigger device has been reset. ON must be set in the case of the activity trigger for the inactivity alarm.	ON

** = Factory setting

*** = Further call types on request.

Technical data

Nominal voltage	24 V=
Standby current consumption	8 mA
Connection cross section	0.14 – 0.5 mm ²
Stripping length	4.5 mm
Minimum switching current of the trigger device	0.1 mA at 5 V DC
LED (reassurance light): - Supply voltage - Maximum power consumption	24 V= 10 mA
Maximum length of cable to trigger device	5 m
Dimensions (HxWxD)	32 x 34 x 16 mm
Degree of protection	IP 20
Ambient temperature	+5°C – +40°C
Relative humidity	0 % – 85 %