**IR TV-Steuermodul universal,
Best.-Nr. 77 0360 11**

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Interface for controlling the TV set with infrared signals. Integrated audio amplifier. Supported TV sets from Philips, Samsung, LG and Grundig see installation instructions for PCM-U "P-Labor Medical Card - Universal".

Scope of delivery:

- Control module
- IR emitter
- Velcro strip
- 3 cable ties, 1 adhesive socket
- Audio cable, 1.50 m
- Network patch cord, 1 m

Configuration

You have to configure the IR TV control module universal as described in the installation instructions for PCM-U "P-Labor Medical Card - Universal". To do this, you need the IR TV installation kit, order no. 77 0360 40; one kit per nurse call system.

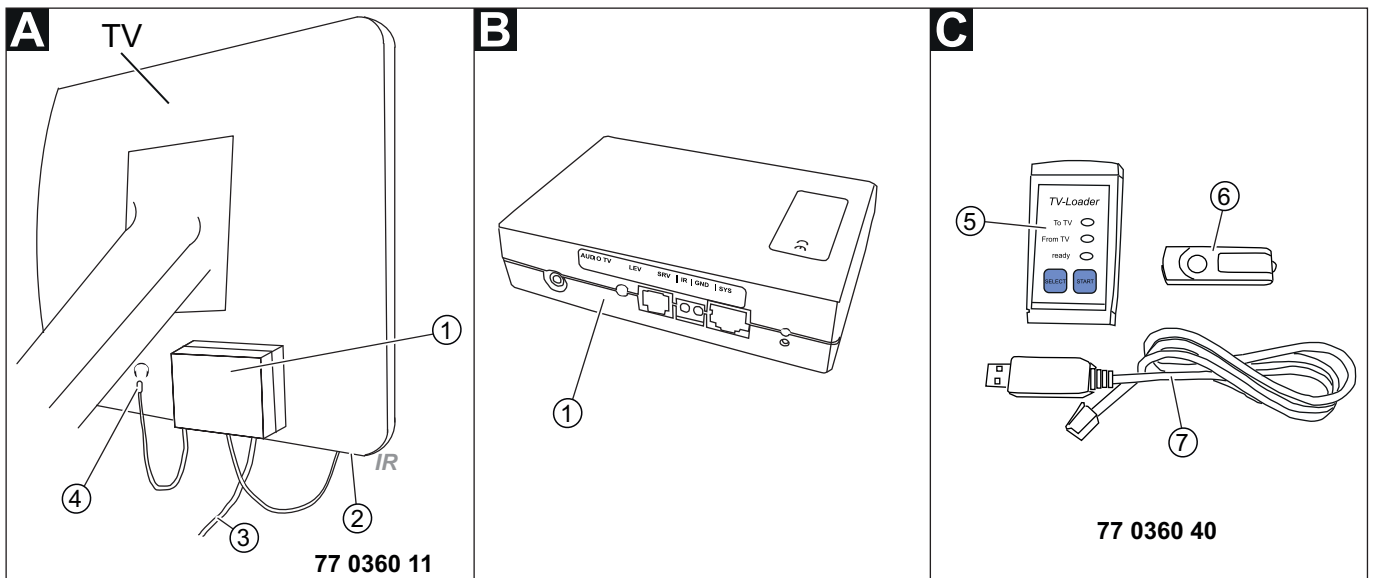
NOTE! The complete installation of the nurse call system is described in the Technical Manual.

CAUTION! The printed circuit board includes electrostatic sensitive components. Avoid touching.

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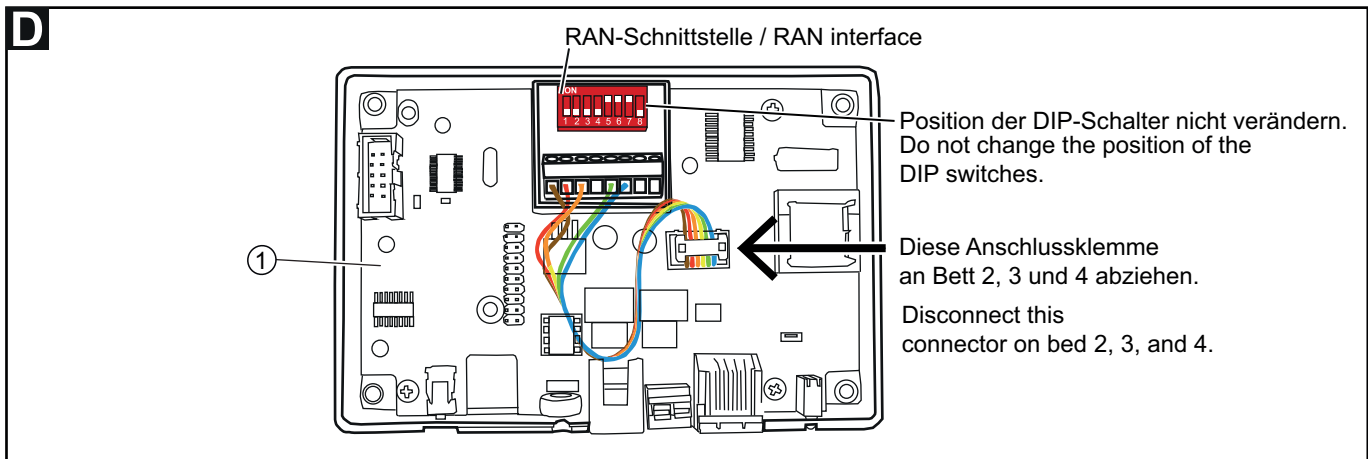
* Scope of delivery of IR TV installation kit (77 0360 40).

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
1. Mounting

1. Each control module[1] has a built-in RAN interface. In an installation with more than one control module per room, however, the RAN interface may only be active on bed 1. This means, open the housings of the control modules of bed 2, 3, and 4 and disable the RAN interface by disconnecting the connector (see fig. D).
2. Connect the cables to the control module[1] as shown in fig. E.
3. Attach the control module to the back of the TV set using the Velcro strip, see fig. A.
4. Affix the IR emitter [2] to the front of the IR receiver of the TV set.
5. Connect the audio cable [4] to the headphone jack of the TV set.
6. Connect the network patch cord to the RJ45 connection socket.
7. Configure the IR TV control module according to the installation instructions for PMC-U "P-Labor Medical Card - Universal".

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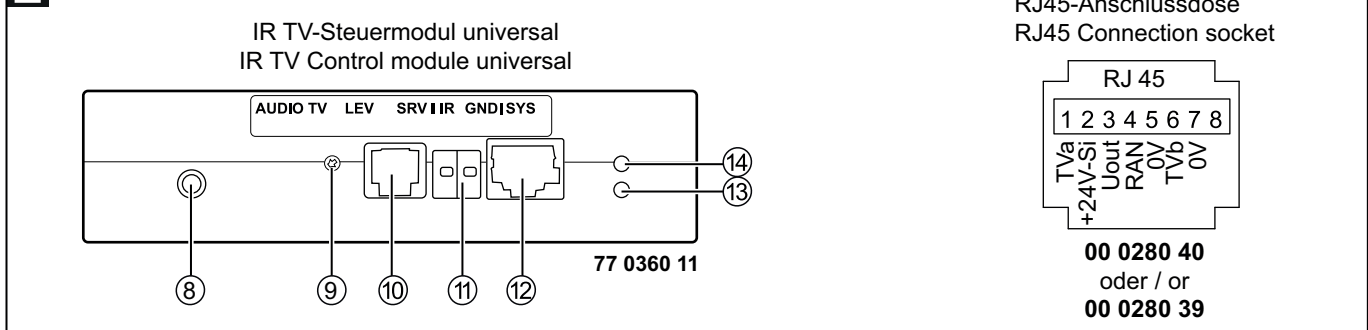
- 8 - Audio cable connection to headphone jack of the TV set
- 9 - Potentiometer for setting the IR emitting power, left: more, right: less.
- 10 - Service connection for temporarily connection of a TV loader or PC
- 11 - IR emitter connection (left: signal, right: ground)

- 12 - Network patch cord connection to RJ45 connection socket
- 13 - Power LED lights up when voltage is applied.
- 14 - Mode LED
 - LED is on: TV set is switched on.
 - LED is off: TV set is switched off.
 - Further signals: Refer to installation guide PMC-U.

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
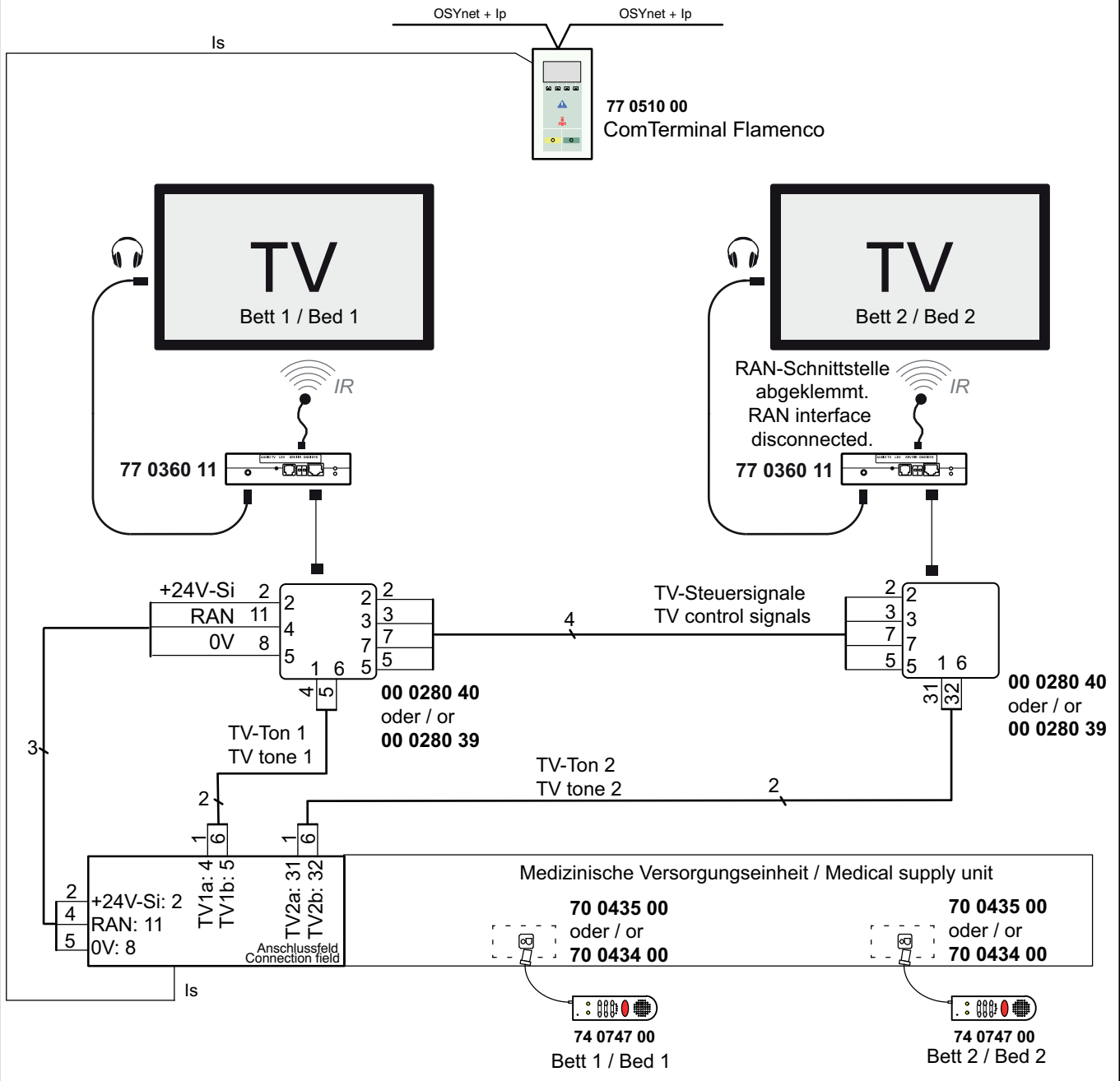
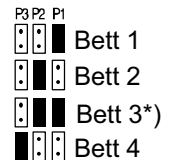
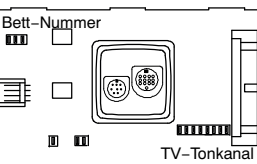
TV-Tonkanal:

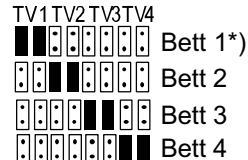
Bed number and TV tone channel

Set the bed number and the TV tone channel by jumpers on the connection socket (70 0435 00 or 70 0434 00):

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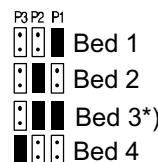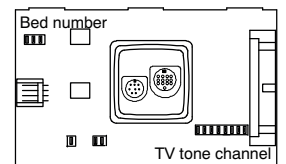
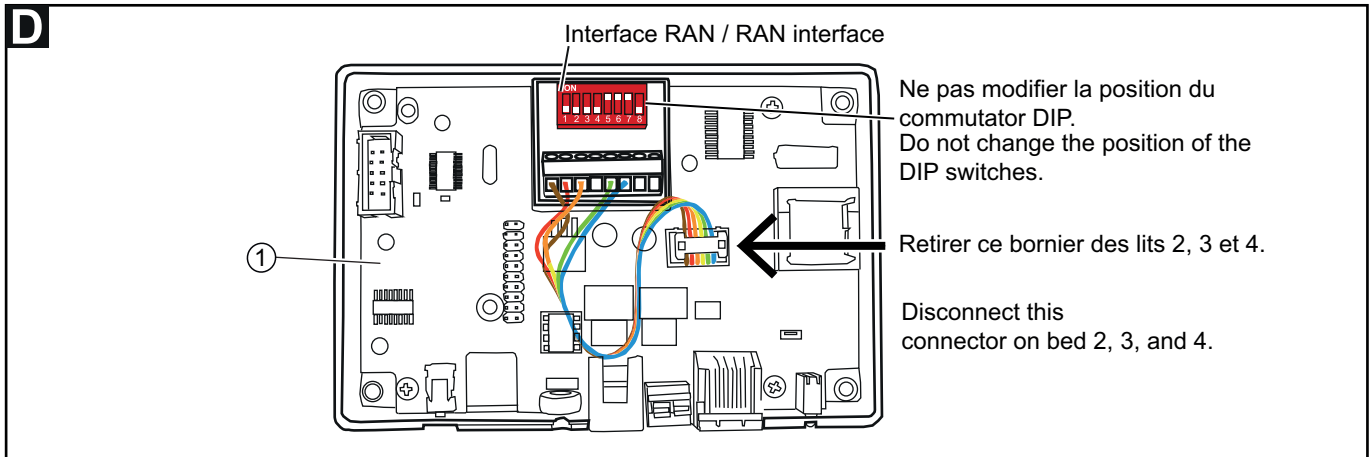
1. Mounting

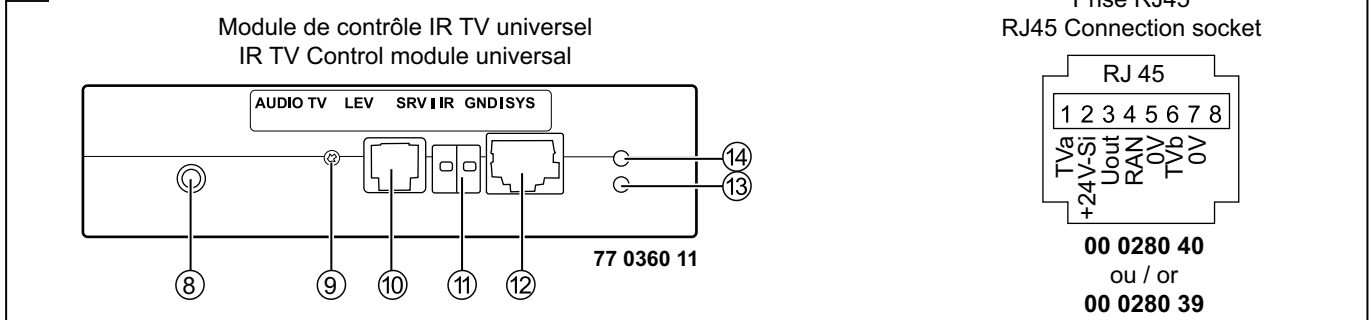1. Each control module[1] has a built-in RAN interface. In an installation with more than one control module per room, however, the RAN interface may only be active on bed 1. This means, open the housings of the control modules of bed 2, 3, and 4 and disable the RAN interface by disconnecting the connector (see fig. **D**).
2. Connect the cables to the control module[1] as shown in fig. **E**.
3. Attach the control module to the back of the TV set using the Velcro strip, see fig. **A**.
4. Affix the IR emitter [2] to the front of the IR receiver of the TV set.
5. Connect the audio cable [4] to the headphone jack of the TV set.
6. Connect the network patch cord to the RJ45 connection socket.
7. Configure the IR TV control module according to the installation instructions for PMC-U "P-Labor Medical Card - Universal".

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© Tunstall GmbH, Orkotten 66, 48291 Teigte, Germany, www.tunstall.de