Zimmerleuchte, 4-teilig, R/A/AW/WC

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**Room lamp, 4 sections, C/A/PR/WC**

- Order no. 77 0176 00: silver design frame
- Order no. 77 0176 01: white design frame

Intended use

Room signal lamp for connection to the room bus RAN of a nurse call system of the Flamenco system family. Four light sections for visual signalling of calls and presence. Installation on walls in corridors and rooms.

LED modules	Colour	Signalling (C/A/PR/WC)	Spare part order no.
	Red	C: Call	77 0190 00
	Blue	A: Cardiac alarm	77 0190 04
	Green	PR: Presence	77 0190 02
	White	WC: WC call	77 0190 03

Signalling

Event	Colour	Fresh call
Call	Red	Standard ¹ : Flashing Option ¹ : Steady light
WC call	Red + White	Standard ¹ : Flashing Option ¹ : Steady light
Emergency call	Red	Rapid flashing
WC emergency call	Red + White	Rapid flashing
Cardiac alarm	Blue	Flickering
Presence	Green	Steady light

¹) The option can be set by DIP switch.

The following applies to all types of call: By answering a fresh call, the flashing changes to a steady light. This means that answered calls light are lit steadily.

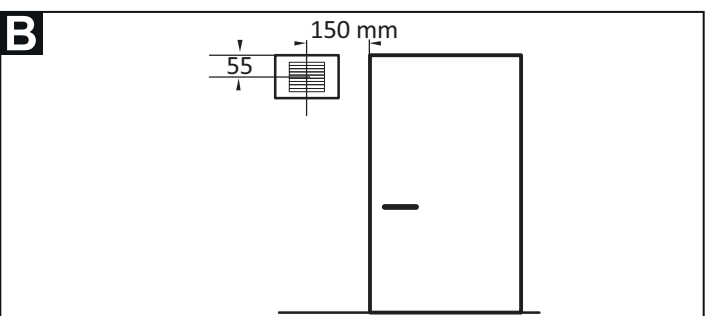
NOTE! The complete installation of the system is described in the Technical Manual.

CAUTION! The printed circuit board and the LED modules include electrostatic sensitive components. Avoid touching.

Defining the mounting position (→ fig. B)

- Mounting on a 1-gang back box (for solid wall: 17 0100 00, for partition wall: 17 5100 00).
- The room lamp must be easily assignable to the room.
- External sources of light must not prevent the room lamp from being identified.
- Mounting height above floor level: 150 – 250 cm.

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Connecting the connector (→ fig. C)

1. Strip the connecting cable in the back box to a suitable length.
2. Remove the cable shield and drain wire up to the cable sheath. Strip the wires. Stripping length: 7 mm.
3. Connect the wires to the connector [3] according to fig. C.

CAUTION! The unused wires must not be connected through.

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Room lamps can be assigned to different channels to form functional units. For the channel number to be set, see the relevant installation plan. Set the channel number as shown in fig. D.

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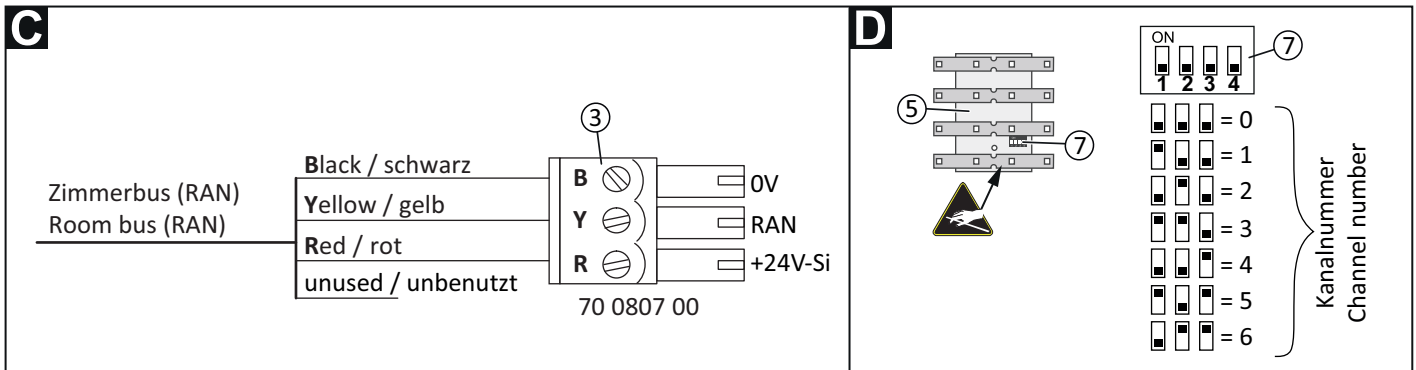
The section “Signalling” describes a standard (factory setting) and an option. You set these with DIP switch 4:

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- DIP switch 4 OFF = Standard (factory setting)
- DIP switch 4 ON = Option

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The ComTerminal in the room has presence buttons for staff 1 (green) and staff 2 (yellow). However, the room lamp only displays presence messages of staff 1. Therefore, the presence button for staff 2 must be disabled via the configuration menu of the ComTerminal.

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Mounting (→ fig. E)

At the point of delivery the room signal lamp is assembled and must be dismantled as follows, see fig. F:

- Lightly compress the light dome [8] from top and bottom. Then pull off the light dome [8].

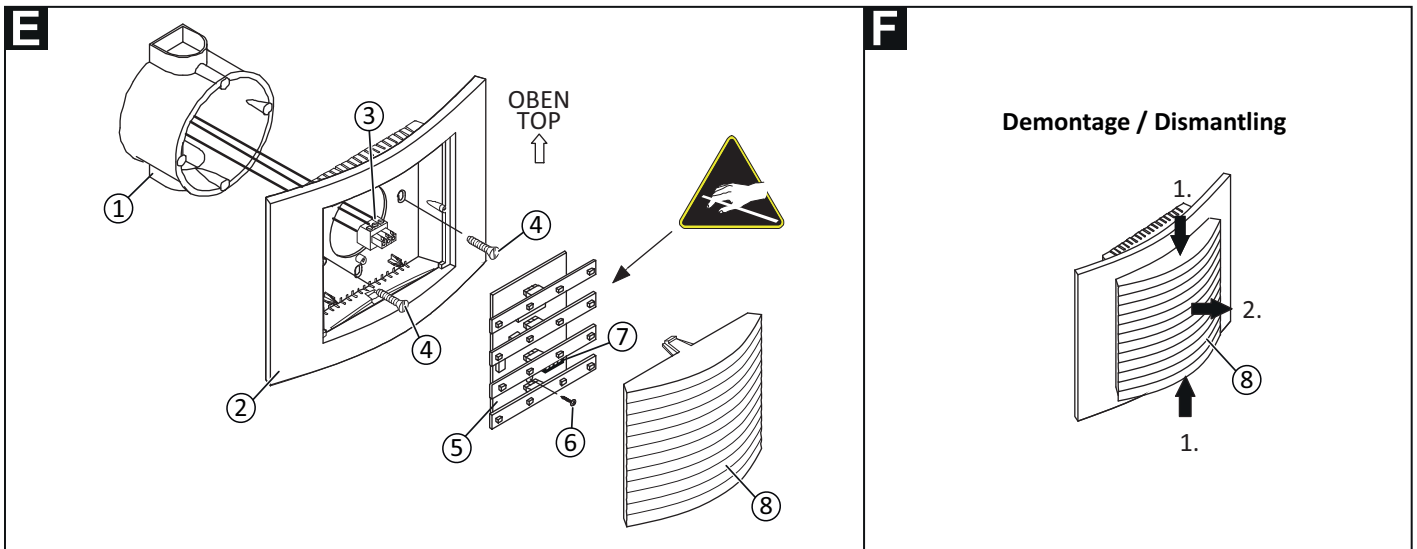
Wall installation on a one-gang back box [1] fixed with screws on the lateral holes of the back box²⁾:

1. Prepare the wires and connect them to the connector [3] (see section “Connecting the connector”).
2. Plug the connector [3] from the rear onto the PCB [5] (in the rear of the housing [2]).
3. Bolt the housing [2] to the back box [1] with the screws [4]. The arrow on the rear of the housing points toward the top.
4. Press the light dome with insert module [8] onto the design frame until it latches in audibly.

²⁾ **NOTE!** To mount the room lamp on the vertical holes of the back box, the PCB must be removed temporarily:

1. Loosen the fastening screw [6] of the PCB and remove the PCB [5] (first bottom, then top).
2. The PCB [5] is installed in the reverse order.

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- | | |
|--------------------------------------|--|
| 1) * Back box, 1-gang | [5] Printed circuit board (PCB) with LED modules |
| [2] Housing | [6] Fastening screw for the PCB |
| [3] * Connector, 3-pole (70 0807 00) | [7] DIP switch |
| [4] * Back box screws | [8] Light dome |

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* Not included in the scope of delivery, please order separately.

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Technical data

Standby current consumption	20 mA
Additional current consumption	30 mA per light section
Standby power consumption	0.48 W
Additional power consumption	0.72 W per light section
Dimensions (HxWxD)	110 x 150 x 40 mm
Housing material	ABS
Light dome material	Styrene-acrylonitrile
Connecting capacity	up to 1.5 mm ²

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© Tunstall GmbH, Orkotten 66, 48291 Telgte, Germany, www.tunstall.de