Display-Einsatz

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Display insert

Intended for connection to the ward bus of a CONCENTO^{CARE} nurse call system. The room lamp and the room bus are connected to the display system. During a failure of the ward bus, the display insert continues to operate independently with the connected devices in the room.

The display insert can be used in combination with an intercom insert for speech communication whose flush-mounted insert is connected to the flush-mounted insert of the display insert.

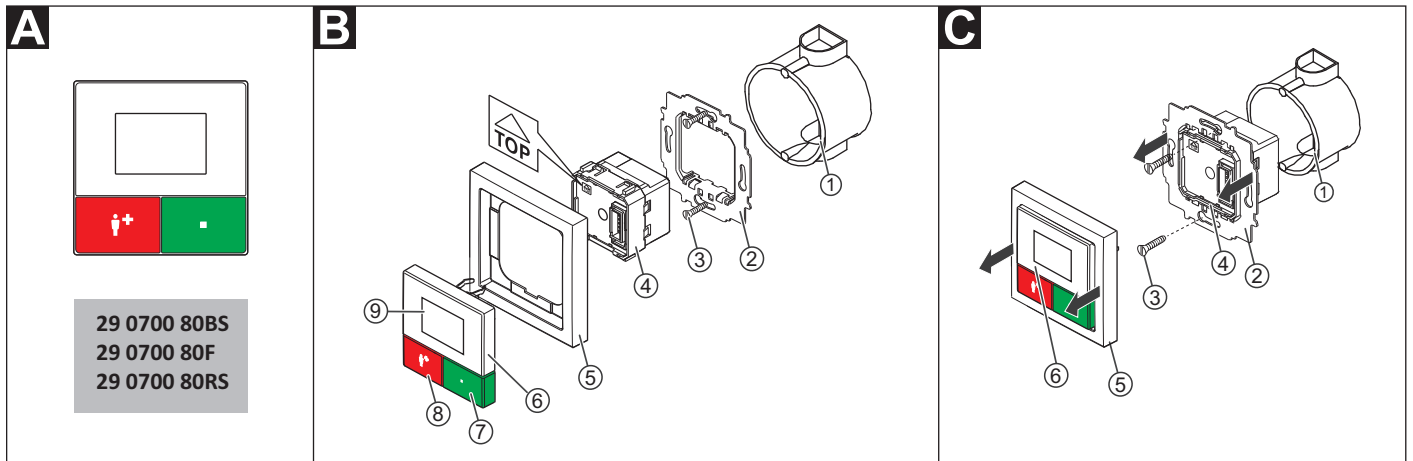
NOTE! The complete installation of the system is described in the Technical Manual.

A Product description

The red call button [8] serves for triggering calls. An LED in the call button lights up faintly to locate the button in the dark (location light). The LED lights up brightly as soon as a call is triggered (reassurance light).

The green presence button [7] serves for switching the presence of the staff on and off. The LED lights up when the presence is switched on (reminder light).

The display [9] and the integrated beeper serves for displaying calls when the presence is switched on. The buttons and the display are additionally used to configure the device.

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- | | |
|---|-----------------------------|
| [1] *Back box, 1-gang
(deep box recommended,
solid wall: 17 0100 20,
partition wall: 17 5100 20) | [4] Basic module |
| [2] Mounting plate | [5] *Cover frame, 1-gang |
| [3] *Back box screws | [6] Control element |
| | [7] Presence button (green) |
| | [8] Call button (red) |
| | [9] Display |

* Not included in the scope of delivery, please order separately.

B Mounting

Wall mounting on one-gang back box.

1. Bolt the mounting plate [2] to the back box [1]. The four square recesses in the mounting plate are located at the top and bottom.
2. Prepare the wires and feed them through the mounting plate [2] (see section "Connection").
3. Connect the basic module [4] according to section "Connection".
4. Push the basic module [4] through the mounting plate [2] until it latches. The mark "TOP" points toward the top.
5. Hold the cover frame [5] in the correct position in front of the mounting plate [2] on the wall.
6. Plug the control element [6] onto the mounting plate with basic module so that the latching hooks of the control element enter the square recesses of the mounting plate and the plug connector of the control element latches into the basic module.

The display insert must first be configured before it is ready for operation, see section "Configuration Instructions".

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C Dismantling

1. Pull the cover frame [5] together with the control element [6] off the mounting plate [2].
2. Loosen the screws [3] with which the mounting plate is fixed to the back box [1].
3. Remove the mounting plate [2] together with the basic module [4] from the back box [1].
4. For disconnection of the wires, refer to fig. E.

Although the basic module [4] remains on the mounting plate [2] during dismantling, it can, however, be separated again in a dismantled state.

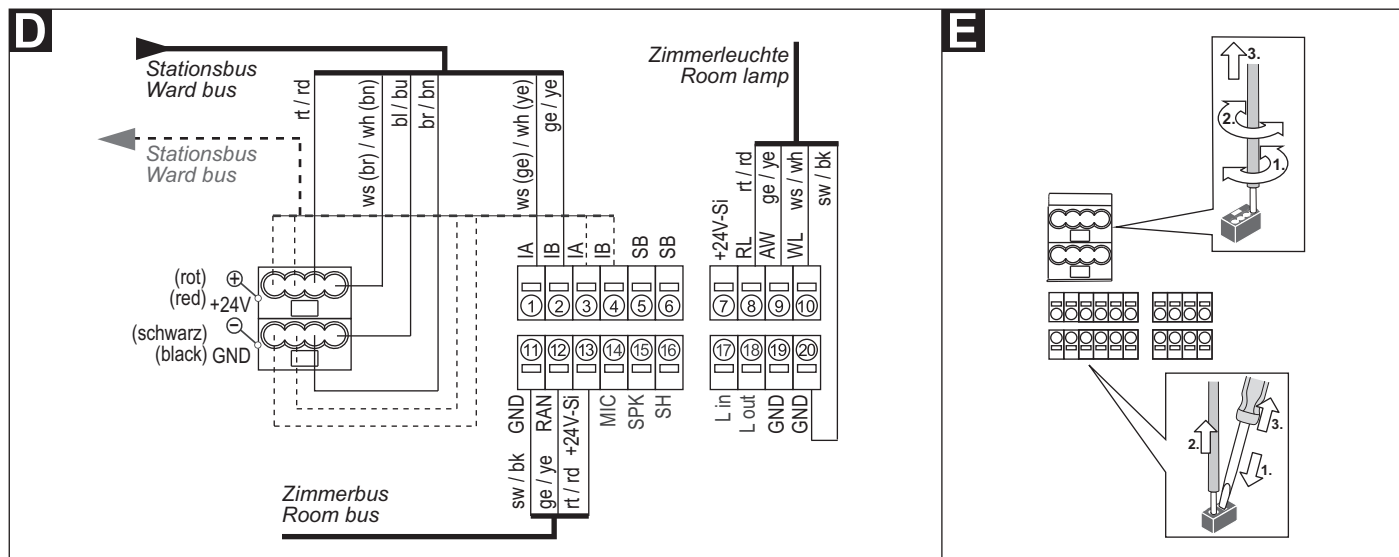
D Connection

1. Strip the connection cables in the back box to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
2. Connect the wires to the lateral terminal block (red, black) and to the connection field on the underside of the basic module according to the connection diagram. All terminals are plug-in terminals.

Note: The lateral terminal block (red, black) can be pulled off temporarily.

3. Bus termination: If the display insert is the first or last user on the ward bus, the bus must be terminated. For this, set a wire bridge between connection points 5 (SB) and 6 (SB).

Note: The connections for the optional intercom insert are described in the installation instructions of the intercom insert.

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EN - Configuration Instructions

The following settings must be made on the display insert:

- Room type,
- The number of devices that are connected to the room bus.

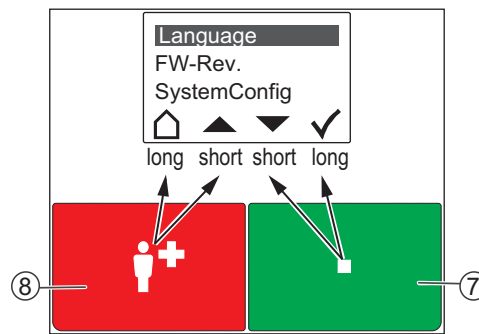
These settings are described in the following. All other settings are carried out with the management software per remote configuration via the bus system and are not part of this document.

i NOTE! As long as the remote configuration has not been carried out, the display shows "WAIT FOR CONFIG". The display insert is not yet ready for operation.

Displays

- 🏠 Quit, i.e. return to the higher menu level without saving.
- ▲ Scroll up the list.
- ▼ Scroll down the list.
- ✓ Set the marked menu item (and save).

Button functions

The red call button [8] and the green presence button [7] each has two functions: One function is triggered with brief press. The other function is triggered with a long press (0.5 seconds).

- The functions of the outer icons in the display are triggered with a long press of the buttons.
- The functions of the inner icons with a brief press.

1 - Calling up the main menu

- Keep the red and green buttons pressed simultaneously (3 seconds) until the main menu is displayed.

2 - Setting the language

1. Use the arrows to go to "Language" in the main menu.
2. Press the **Tick**.
3. Use the arrows to move to the desired language.
4. Press the **Tick** to save the marked language.

Note: During the remote configuration, the language that has been set is automatically changed by the management software to the selected language of the management software.

3 - Calling up the service menu and performing settings

1. Use the arrows to go to "SystemConfig" in the main menu.
2. Press the **Tick**. A password is requested.
3. Press the arrows in the following order as password:
▲▲▼▲▼▼

Setting the room type

i NOTE! Room type "Staff+HAB" can only be set if the nurse call system has already been set to "Manual coupling" with the management software. Only one room per nurse call system is permitted to be set as "Staff+HAB"!

1. Use the arrows to go to "Room Type" in the service menu.
2. Press the **Tick**.

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EN - Configuration Instructions

3. Use the arrows to move to the desired room type:
 - "Room" = Residents room (factory setting)
 - "Staff" = like "Room" + announcement option (only with connected intercom insert).
 - "Staff+HAB" = like "Staff" + option of switching over time zones.
4. Press the **Tick** to save the marked room type.

Setting the number devices on the room bus

Devices on the room bus = Switch inserts, call switches with connection socket, RAN interfaces, intercom insert.

i NOTE! Also the intercom insert counts as a device on the room bus and must be included in the count. The room lamp is not a device on the room bus!

Factory setting = 1 device on the room bus!

1. Use the arrows to go to "RAN" in the service menu.
2. Press the **Tick**.
3. Use the arrows to go to "RAN number".
4. Press the **Tick**.
5. Use the arrows to move to the correct number of devices on the room bus.
6. Press the **Tick** to save the marked number.

4 - Testing the room bus

The following functions serve to test the room bus:

Testing the status

Check whether the set number of devices on the room bus matches the number of devices on the room bus that are ready for operation.

1. Use the arrows to go to "RAN" in the service menu.
2. Press the **Tick**.
 - Display of "Status OK" = The set number matches the number of devices on the room bus that are ready for operation.
 - Display of "Status ERROR(+)" = More devices ready for operation were recognized on the room bus than have been set.
 - Display of "Status ERROR(-)" = Less devices ready for operation were recognized on the room bus than have been set.
3. Press **House** to quit the display.

If a fault (ERROR) has been displayed, carry out the test as follows:

Performing the test

Test whether all devices on the room bus have been connected correctly and are ready for operation:

1. Use the arrows to go to "RAN" in the service menu.
2. Press the **Tick**.
3. Use the arrows to go to "RAN test".
4. Press the **Tick**.
 - The LEDs of all devices on the room bus must flash. Devices not flashing have been connected incorrectly or are defective.
 - Replace defective devices.
 - Connect devices correctly that have not been connected correctly.
5. Press **House** to quit the service menu.

5 - Quitting the configuration

To terminate the configuration, quit the service menu and the main menu.

- Keep pressing **House** until the normal operating display appears.

Note: If no button is pressed within a period of 60 seconds, the menu is quit automatically.