## Intercom-Einsatz

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## Intercom insert

The intercom insert is intended for connection to a room control device, i.e. a display insert or a call/presence insert in a CONCENTO<sup>CARE</sup> nurse call system. The intercom insert supplements the room control with the speech communication function. For the installation with display insert, additionally for announcements.

**NOTE!** The complete installation of the system is described in the Technical Manual.

- |   |   |
|---|---|
| [1] *Back box, 1-gang, for the room control device (deep box recommended)   | [7] *Basic module of room control   |
| [2] *Back box, 1-gang, for the intercom insert (deep box recommended, solid wall: 17 0100 20, partition wall: 17 5100 20) | [8] *Cover frame, 2-gang  |
| [3] *Mounting plate of the room control device  | [9] *Control element of the room control device   |
| [4] Mounting plate of the intercom insert   | [10] Intercom insert  |
| [5] *Screws of the back box for the room control device   | [11] Answer button (Icon: loudspeaker) at activated presence for the answering of calls |
| [6] *Screws of the back box for the intercom insert   | [12] Remote cancel button (Icon: 2 arrows) for the remote cancelling of answered calls  |
|   | [13] Microphone and loudspeaker for speech communication                                |
|   | [14] External terminal block (black, yellow, red) for the room bus                      |

\* Not included in the scope of delivery, please order separately.

### B Mounting

Wall installation on two connected one-gang back boxes:

- Bolt mounting plates [3] [4] butt-jointed on the back boxes [1] [2]. The four square recesses in the mounting plate are located at the top and bottom.
- Prepare and connect the wires in the back box for the room control device [1] (see section D).
- Push the basic module [7] through the mounting plate [3] until it latches. The mark "TOP" points toward the top.
- Position and hold the cover frame [8] in the correct position before the two mounting plates.
- Plug the control element [9] onto the mounting plate with basic module so that the latching hook of the control element enters the square recesses of the mounting plate and the plug connector of the control element latches into the basic module.
- Connect the wires for the intercom insert [10] (see section E).
- Plug the intercom insert [10] onto the mounting plate [4] so that the latching hook of the intercom insert fits into the square recesses of the mounting plate.

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**C Dismantling**

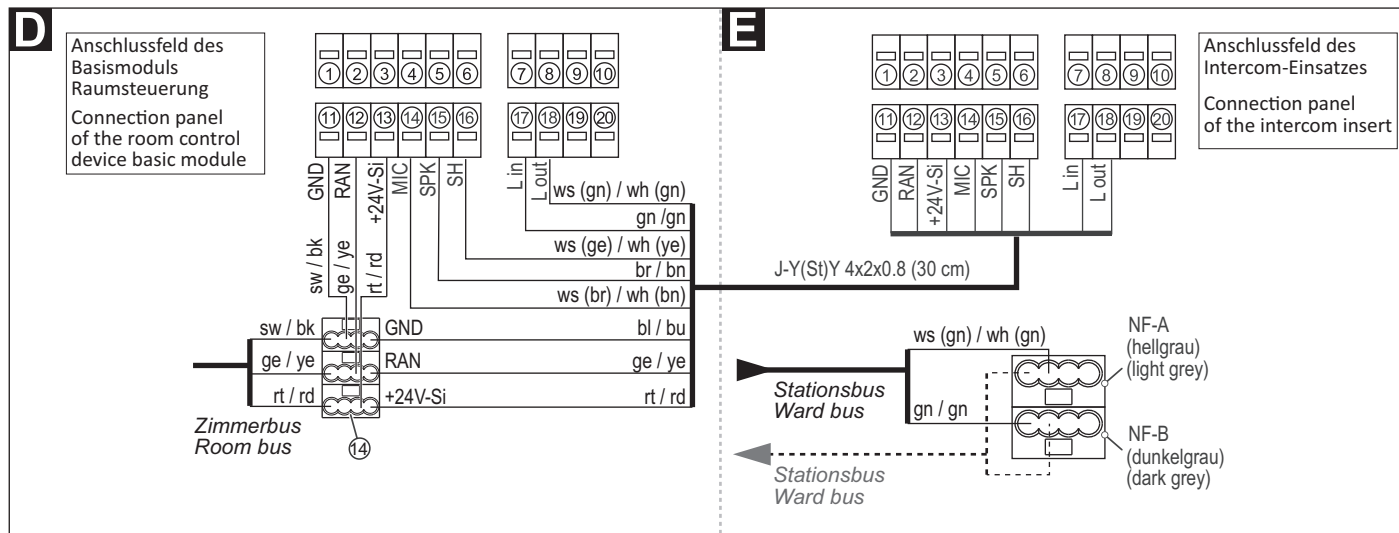
1. Pull the cover frame [8], the control element of the room control [9] and the intercom insert [10] off together. To do this, grip the lower third of the control element [9]. If the cover frame does not come loose, carefully lever it off with a screwdriver.
2. Loosen the screws [5] with which the mounting plate [3] of the room control is fixed to the back box [1].
3. Remove the mounting plate [3] together with the basic module [7]. Although the basic module [7] remains connected to the mounting plate after dismantling, it can, however, be separated again in a dismantled state.
4. Loosen the screws with which the mounting plate [4] of the intercom insert is fixed to the flush-mounted box.
5. Remove the mounting plate [4].

**D Connection of basic module of room control**

1. Connect the ward bus (up to NF-A and NF-B) and the room lamp to the basic module of the room control according to the installation instructions of the room control.
2. Install wires NF-A and NF-B from the ward bus through the connection of the back box to the intercom insert.
3. Connect the connecting points 11 – 18 of the connection panel according to fig. D to a J-Y (ST)Y 4x2x0.8 cable approx. 30 cm long.
4. Connect the room bus to the external terminal block.
5. Install cable J-Y (ST)Y 4x2x0.8 through the connection of the back boxes to the intercom insert.

**E Connection of intercom insert**

1. Connect wires NF-A and NF-B from the ward bus to the lateral terminal block (light grey, dark grey) of the intercom insert. The terminal block can be pulled off temporarily.
2. Connect connecting points 11 - 18 of the connection panel of the room control (fig. D) 1:1 with connecting points 11 - 18 of the connection field on the underside of the intercom insert (fig. E).

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