Zugtaster-Einsatz

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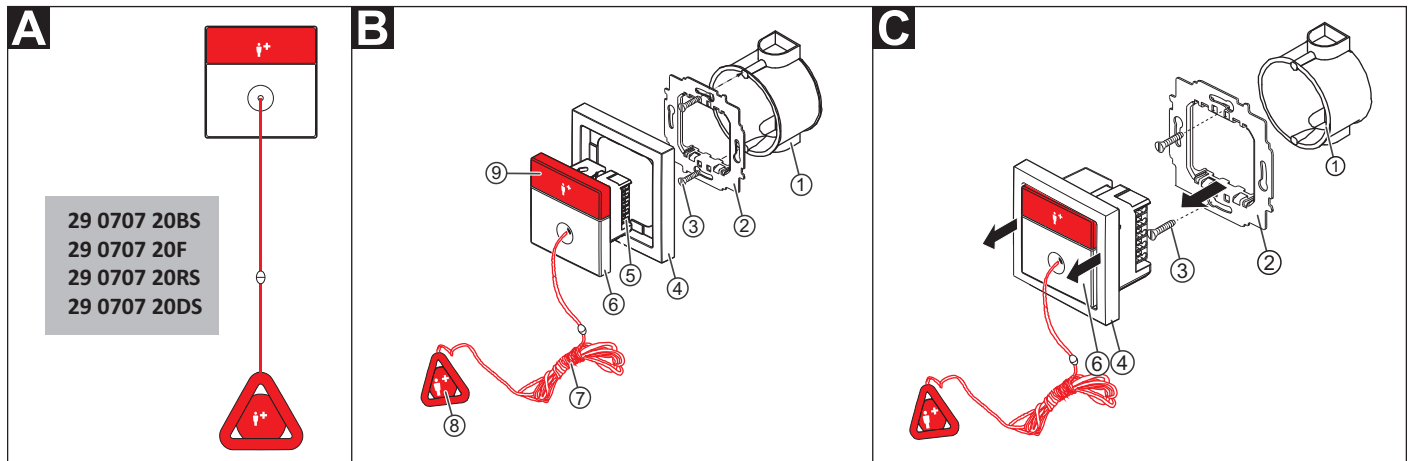
WC calls are cancelled in the standard case by pressing a grey cancel button at the call location. The pull-cord switch can be configured in such a way that WC calls can not only be triggered with the pull cord [7] but also cancelled if presence is switched on.

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- [1] *Back box, 1-gang
(solid wall: 17 0100 00,
partition wall: 17 5100 00)
- [2] Mounting plate
- [3] *Back box screws
- [4] *Cover frame, 1-gang

- [5] Terminal block
(plug-in terminals)
- [6] Pull cord switch insert
- [7] Pull cord (2.50 m) incl. safety release
- [8] Call handle
- [9] Call button (red)

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1. Bolt the mounting plate [2] to the back box [1]. The four square recesses in the mounting plate are located at the top and bottom.
2. Prepare the wires and feed them through the mounting plate (see section "Connection").
3. Connect the pull cord switch insert [6] according to section "Connection".
4. Hold the cover frame [4] in front of the mounting plate on the wall.
5. Plug the pull cord switch insert [6] onto the mounting plate so that the latching hook of the pull cord switch insert fits into the square recesses of the mounting plate. The call button is located at the top.
6. The call handle [8] must be located between 10 and 20 cm above the floor. The safety release must be located above the position of the pull cord where it will be pulled to trigger a call. To maintain these characteristics of the pull cord [7], shorten the pull cord [7] at the safety release or in the call handle and re-knot it there again.

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Configuration

The following functions can be configured if required:

Channel number (1 – 6)

A channel number can be set on the pull cord switch to specify the call location in the room in more detail. The channel number can, for example, identify beds or WC cubicles.

- Factory setting = channel 0, i.e. no channel set.

WC call cancel function

The WC call cancel function is available from production week 24 in 2021. The production week can be found in the serial number on the unit: XXXXXXXX-XX-2421-XXXXX.

If the pull cord switch is set to "WC call" (= wire bridge between LT1 and LT2), you can configure the pull cord switch so that WC calls (and WC emergency calls) can not only be triggered but also cancelled with the pull cord.

CAUTION! The channel number also applies to the WC call cancel function. If a channel number is set, WC calls that were triggered with this pull cord switch or another call switch with the same channel number can be cancelled using this pull cord switch. WC calls from all channel numbers can be cancelled with a pull cord switch set to channel 0 (no channel).

NOTE! To comply with the German standard DIN VDE 0834, an additional cancel switch must be installed.

- Factory setting = WC call cancel function is disabled.

How to proceed to the configuration

CAUTION! When configuring, both the channel number (0 – 6) and the WC call cancel function (on or off) are always configured. It is not possible to configure only one parameter.

For configuring the pull cord switch within the first ten seconds after the power supply is switched on, start with the following procedure. (Alternatively, the terminal block [5] can be pulled off briefly and then re-plugged.)

1. Press and hold the button [9] for 5 seconds until a triple tone sounds. (During the 5 seconds, the LED must be off).

The LED in the button flashes according to the channel number set: Channel 1 = once, Channel 2 = twice and so on, Channel 0 (no channel) = LED lights up for 2 seconds.

2. Set the channel number with the button as follows: Channel 1 = press 1 time briefly, Channel 2 = press 2 times briefly, etc. To set channel 0 (no channel), do not press the button.
3. Configure the WC call cancel function with the pull cord as follows: Enable = pull 1 time (short tone is emitted). Disable = do not pull the pull cord.
4. To save both settings, press and hold the button again for 5 seconds until a tone sounds.

The LED in the button flashes according to the channel number set: Channel 1 = once, Channel 2 = twice and so on, Channel 0 (no channel) = LED lights up for 2 seconds.

Timeout: If the button is not pressed within 30 seconds, a tone sounds and the call pull switch exits configuration mode. The new settings are not saved.

Function test: After configuration, be sure to perform a function test of the configured functions.

Function test of the WC call cancel function

With the WC call cancel function, WC calls and WC emergency calls can be cancelled when presence is switched on:

1. Trigger a WC call with the pull cord switch.
2. Press a presence button in the room.
3. Hold the pull cord pulled for 5 seconds until a triple tone sounds and then release.

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Pull cord switch insert

Intended for connection to the room bus (RAN) of a Tunstall nurse call system.

NOTE! The complete installation of the system is described in the Technical Manual.

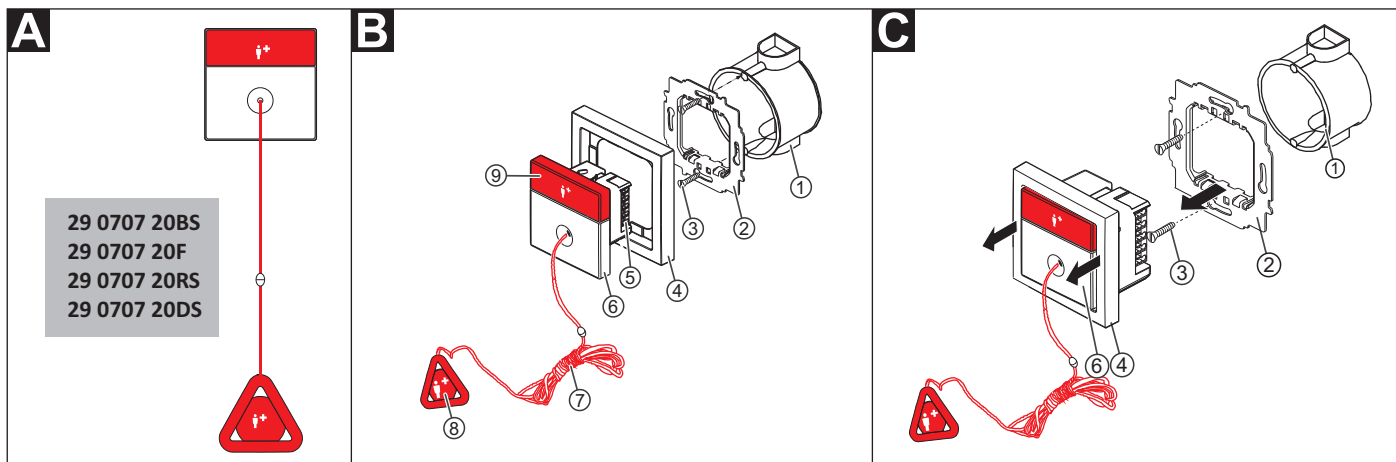
A Product description

The pull cord switch serves for triggering calls or WC calls. A pull cord [7] and a red call button [9] serve for triggering calls. An LED in the call button [9] lights up faintly to locate the button in the dark (location light). The LED lights up brightly as soon as a call is triggered (reassurance light). If presence in the room is switched on when the call is triggered, an emergency call is triggered.

A safety release in the pull cord [7] opens when the pulling force exceeds a limit value of approx. 25 Nm. The safety release is easily closed again by simply pushing it together.

As an option: WC call cancel function

WC calls are cancelled in the standard case by pressing a grey cancel button at the call location. The pull-cord button can be configured in such a way that WC calls can not only be triggered with the pull cord [7] but also cancelled if presence is switched on.

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- | | |
|--|---|
| [1] *Back box, 1-gang
(solid wall: 17 0100 00,
partition wall: 17 5100 00) | [5] Terminal block
(plug-in terminals) |
| [2] Mounting plate | [6] Pull cord switch insert |
| [3] *Back box screws | [7] Pull cord (2.50 m) incl. safety
release) |
| [4] *Cover frame, 1-gang | [8] Call handle |
| | [9] Call button (red) |

* Not included in the scope of delivery, please order separately.

B Mounting

CAUTION! Pull cord switches in shower stalls must be installed at least 20 cm above the highest possible position of the shower head.

Ceiling or wall installation on one-gang back box:

1. Bolt the mounting plate [2] to the back box [1]. The four square recesses in the mounting plate are located at the top and bottom.
2. Prepare the wires and feed them through the mounting plate (see section "Connection").
3. Connect the pull cord switch insert [6] according to section "Connection".
4. Hold the cover frame [4] in front of the mounting plate on the wall.
5. Plug the pull cord switch insert [6] onto the mounting plate so that the latching hook of the pull cord switch insert fits into the square recesses of the mounting plate. The call button is located at the top.

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EN - Installation Instructions

6. The call handle [8] must be located between 10 and 20 cm above the floor. The safety release must be located above the position of the pull cord where it will be pulled to trigger a call. To maintain these characteristics of the pull cord [7], shorten the pull cord [7] at the safety release or in the call handle and re-knot it there again.

C Dismantling

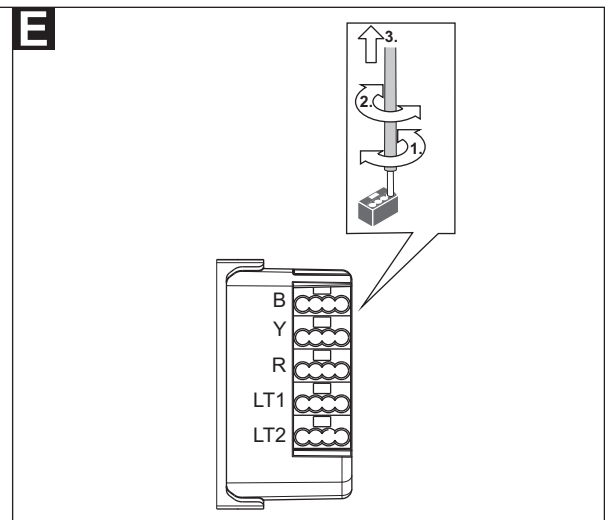
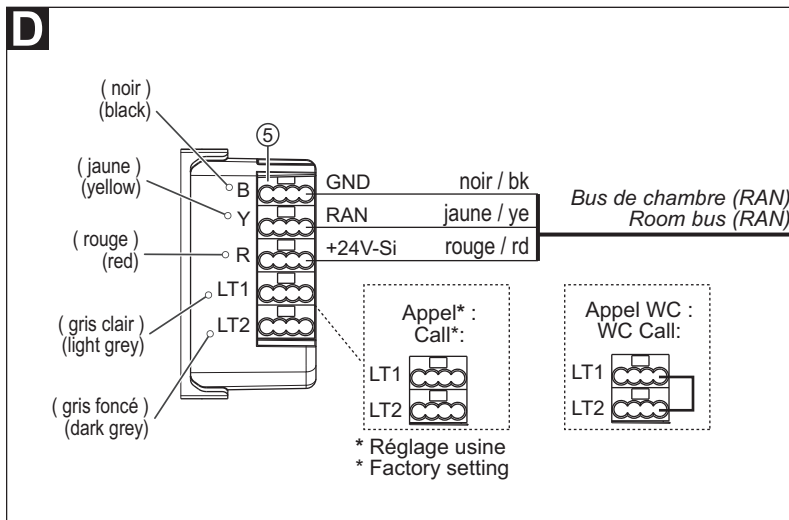
1. Pull the cover frame [4] together with the pull cord switch insert [6] off the mounting plate [2].
2. For disconnecting the wires, refer to fig. E.

D Connection

1. Pull the connection cable out through the mounting plate and strip it to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
2. Connect the wires according to the connection diagram.

NOTE: The terminal block [5] can be pulled off temporarily.

3. If the pull cord switch insert is to be used for triggering WC calls, a wire bridge must be fitted between connection points LT1 and LT2. No wire bridge is to be set for triggering standard calls (= factory setting).

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For configuring the pull cord switch within the first ten seconds after the power supply is switched on, start with the following procedure. (Alternatively, the terminal block [5] can be pulled off briefly and then re-plugged).

1. Press and hold the button [9] for 5 seconds until a triple tone sounds. (During the 5 seconds, the LED must be off).
The LED in the button flashes according to the channel number set: Channel 1 = once, Channel 2 = twice and so on, Channel 0 (no channel) = LED lights up for 2 seconds.
2. Set the channel number with the button as follows: Channel 1 = press 1 time briefly, Channel 2 = press 2 times briefly, etc. To set channel 0 (no channel), do not press the button.
3. Configure the WC call cancel function with the pull cord as follows: Enable = pull 1 time (short tone is emitted). Disable = do not pull the pull cord.
4. To save both settings, press and hold the button again for 5 seconds until a tone sounds.

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