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Call switch insert with connector

Intended for connection to the room bus of a CONCENTO^{CARE} nurse call system.

The connector is intended for the connection of a pear push switch, or order no. 29 0790 xx. Pear push switches serve for triggering calls. Pear push switches incl. light switches can also be used to switch light, either with or without dimming function.

NOTE! The complete installation of the system is described in the Technical Manual.

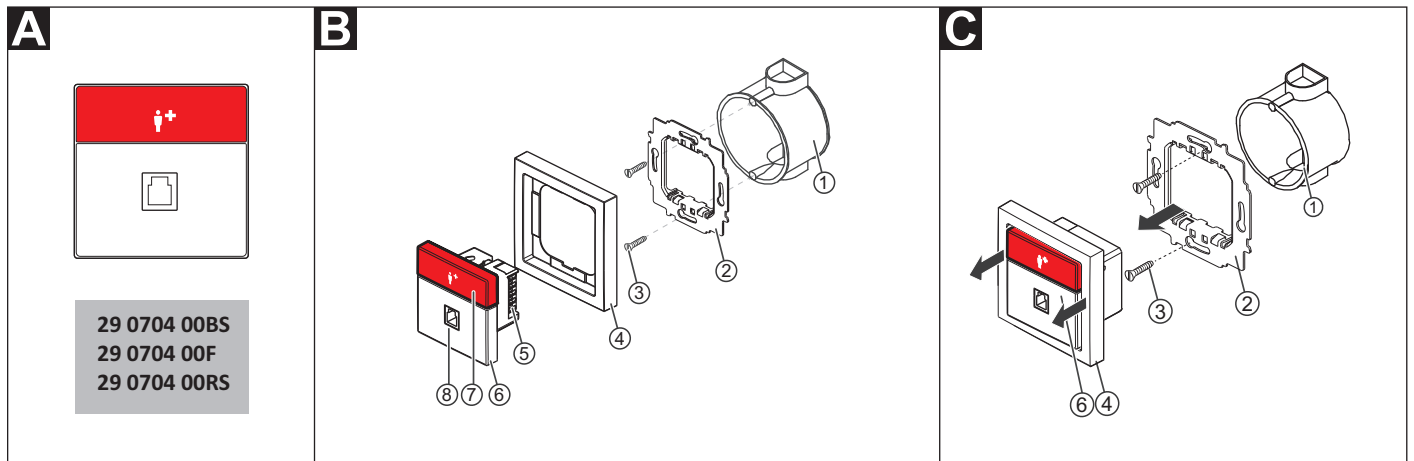
A Product description

The red call button [7] serves for triggering calls. An LED in the call button lights up faintly to locate the button in the dark (location light). The LED lights up brightly as soon as a call is triggered (reassurance light).

Light activation

The call switch insert with connector has two outputs for the activation of lights (LT1 and LT2). If both outputs are used, LT1 is intended for reading light and LT2 for room light. The outputs output a 24 V= signal for the duration the light button is pressed.

Note: Pear push switch, order no. 29 0790 02, only uses LT1. This can be used either for reading light or room light.

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* Not included in the scope of delivery, please order separately.

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B Mounting

Wall installation on one-gang back box:

1. Bolt the mounting plate [2] to the back box [1]. The four square recesses in the mounting plate are located at the top and bottom.
2. Prepare the wires and feed them through the mounting plate [2] (see section "Connection").
3. Connect the call switch insert [6] according to section "Connection".
4. Hold the cover frame [4] in front of the mounting plate [2] on the wall.
5. Plug the call switch insert [6] onto the mounting plate so that the latching hook of the call switch insert fits into the square recesses of the mounting plate [2].

C Dismantling

1. Pull the cover frame [4] together with the call switch insert [6] off the mounting plate [2].
2. For disconnecting the wires, refer to fig. E.

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D Connection

1. Pull the connection cable out through the mounting plate and strip it to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
2. Connect the wires according to the connection diagram.

Note: Pear push switch (29 0790 02) uses LT1 only!

Pear push switch (29 0790 00) does not use LT1 or LT2.

Note: The terminal block [5] can be pulled off temporarily.

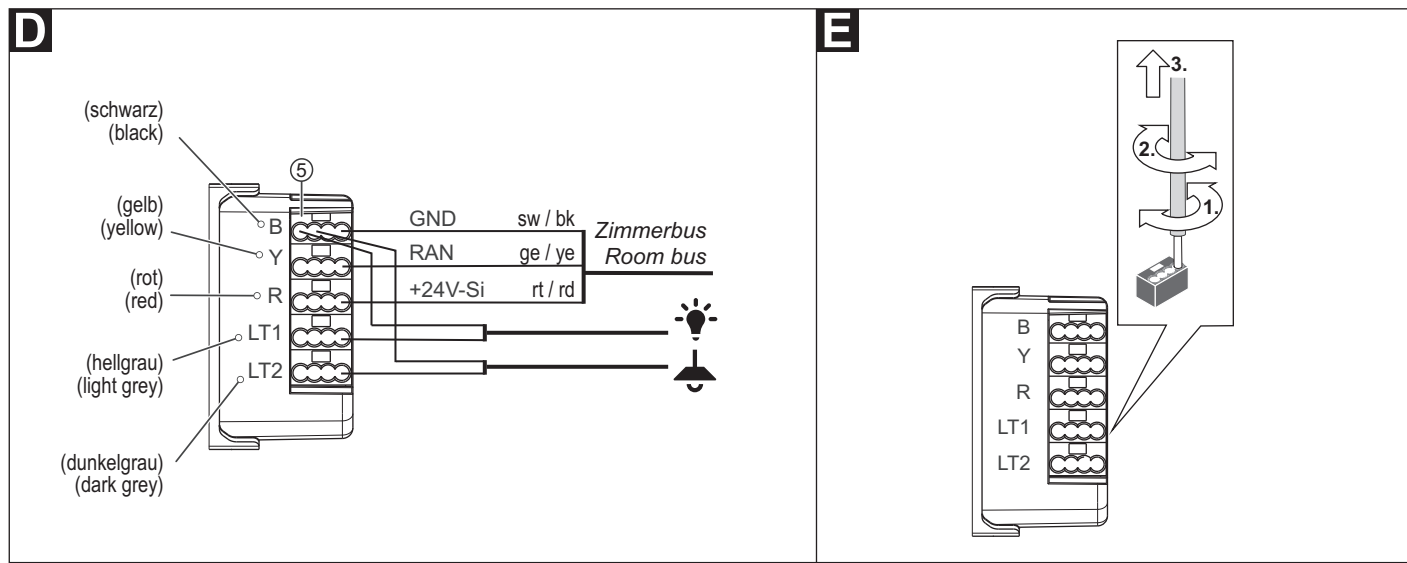
Setting the bed number (1 – 6)

To specify the call location more accurately in a room with multiple beds, the bed number can be set (bed 1 to 6).

- Factory setting = Bed 1.

For setting a bed number within the first ten seconds after the power supply is switched on, start with the procedure described in the following. (Alternatively, the terminal block [5] can be pulled off briefly and then re-plugged).

1. Keep the call button [7] pressed for 5 seconds.
- ✓ The LED on the call button flashes according to the bed number set: Bed 1 = once, Bed 2 = twice and so on, if no bed is set = LED lights up for 2 seconds.
2. To change the bed number, briefly press the call button [7]: Bed 1 = once, Bed 2 = twice and so on. If "no bed number" is to be set, do not press the call button.
3. To save the setting, again press the call button [7] for 5 seconds. Note: If you do not keep the call button pressed, the setting is not saved.
- ✓ The LED on the call button flashes according to the bed number set: Bed 1 = once, Bed 2 = twice and so on, if no bed is set = LED lights up for 2 seconds.

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