## Gruppencontroller

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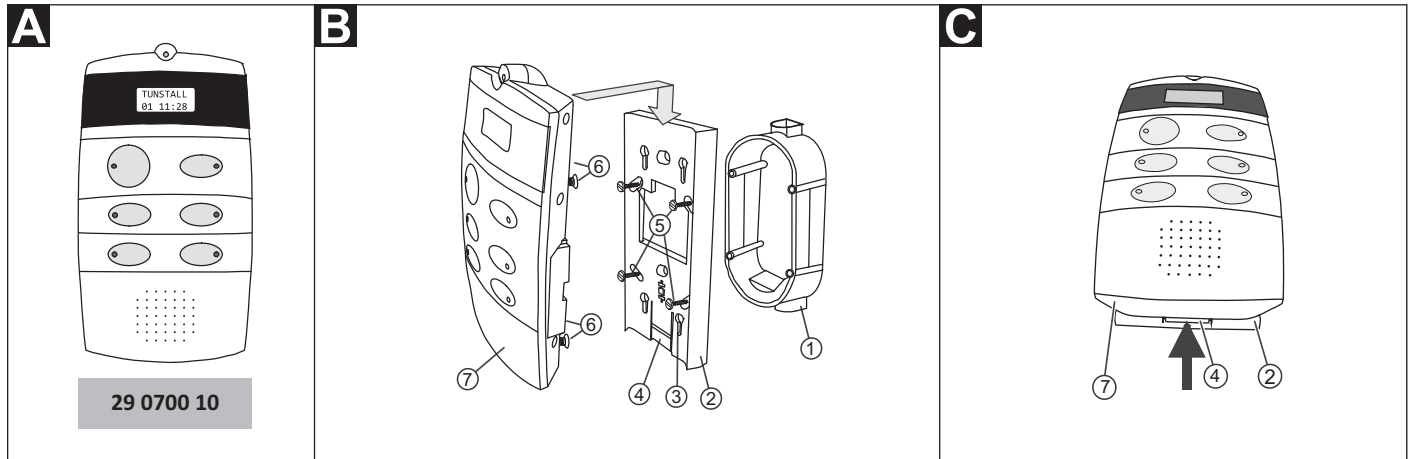
**NOTE!** The complete installation of the system is described in the Technical Manual.

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[1] \*Back box, 2-gang  
(solid wall: 17 0410 00,  
partition wall: 17 5400 00)

[2] Mounting plate  
[3] Keyholes for attaching the  
group controller

[4] Latching hook of mounting  
plate

[5] \*Four screws of the back box  
[6] Four projecting screws  
[7] Group controller

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\* Not included in the scope of delivery, please order separately.

Additionally there are 3 screw-type terminals included in the scope of supply: 4-pole connector, 10-pole connector, 12-pole connector.

### B Mounting

Wall installation on 2-gang back box:

1. Prepare the wires and connect them according to section "Connection". Set the DIP switch according to section "Connection".
2. Feed the three connected connectors through the opening of the mounting plate [2].
3. Fix the mounting plate [2] to the back box [1] with the four screws of the back box [5].
4. Plug the three connected connectors onto the connection panel of the group controller [7].
5. Set the group controller [7] onto the mounting plate [2] so that the 4 projecting screws [6] are inserted into the keyholes [3] of the mounting plate.
6. Push the group controller [7] down until it latches in audibly.

**Note:** Before the group controller is ready for operation, it must be remote-configured with the management software via the bus system. As long as the remote configuration has not been carried out, the display shows "WAIT FOR CONFIG". The LED in button 2 (top right) below the display lights up. The fault message relay has become operative.

### C Dismantling

1. Press the latching hook [4] on the underside of the group controller in the direction of the wall with the thumb of one hand to loosen the lock. At the same time push up the group controller [7] 1 cm with the other hand and remove it from the mounting plate [2].
2. Pull off the connectors from the connection panel on the rear of the group controller.

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## EN - Installation Instructions

### D E Connection

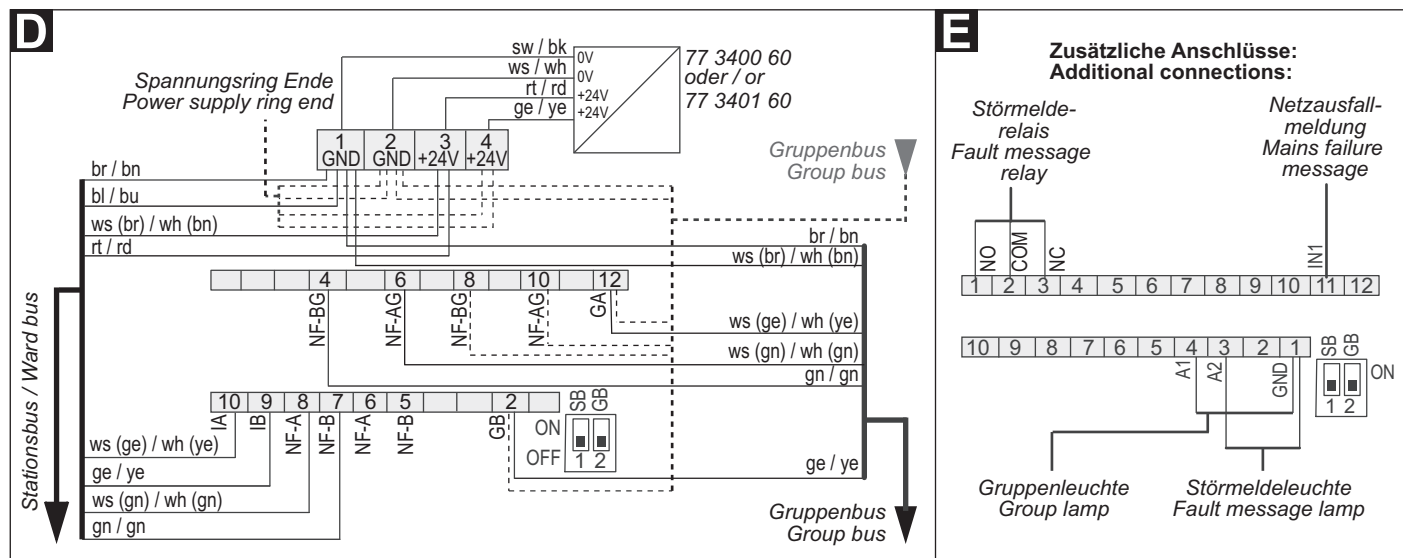
- Strip the connection cables in the back box to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
- Connect the wires to the three connectors according to the connection diagrams **D** and **E**.

**Note about the 4-pole connector:** No bridge required in the plug between 1 and 2 (GND) and 3 and 4 (+24 V). Bridges are on the circuit board.

**Note about the group bus and ward bus:** NF-AG, NF-BG, NF-A and NF-B are not used in systems without speech communication.

### Information about the additional inputs/outputs

<b>Fault message relay</b>	Potential-free. Switching capacity: 2 A; Switches at: Fault at group controller, group controller not fully programmed. NO = Normally open contact, NC = Normally closed contact.
<b>Mains failure message</b>	From the fault message contact of the power supply unit.
<b>Group lamp</b>	Load capacity of output 300 mA. Switches during: Calls, presence and faults on the ward bus.
<b>Fault message lamp</b>	Load capacity of output 300 mA. Switches during: Fault on the ward bus.

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- The two DIP switches on the rear of the group controller serve for setting the bus terminations for group bus and ward bus:
  - If the group controller is the first or last user on the ward bus, set the DIP switch 1 (SB) to position ON.
  - If the group controller is the first or last user on the group bus, set the DIP switch 2 (GB) to position ON.

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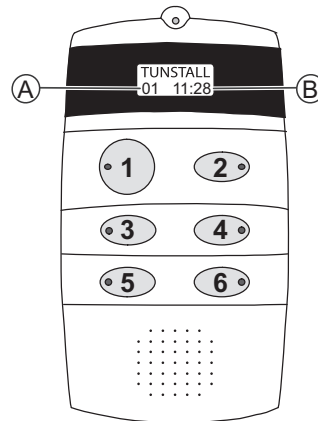
## Displays during operation

- To switch on the display for 10 seconds, briefly press **button 1** or **2**.  
✓ The following is displayed: [A] = Number of active messages (calls, presences, faults) on the ward bus, [B] = Time
- Press **button 4** to scroll through the details of the active messages (calls, presences, faults).

## LED displays during operation

Button 1: Function of the group controller on the group bus	
LED 1 lights up permanently.	Group controller is the master.
LED 1 is off.	Group controller is a slave.
LED 1 flashes (4 s on / 4 s off).	The group controller is the only group bus user or is not connected to the group bus.
Button 2: Operating status of the group controller	
LED 2 is off.	Group controller is ok.
LED 2 lights up permanently.	Group controller is faulty or not fully configured.
Button 3, Button 4	
	Evaluation only by Tunstall.
Button 5: Polling the group bus	
LED 5 lights up permanently or is off.	Fault on the group bus.
LED 5 flashes rhythmically	Group bus is ok.
Button 6: Polling the ward bus	
LED 6 lights up permanently or is off.	Fault on the ward bus.
LED 6 flashes rhythmically.	Ward bus is ok.

## Service menu

The service menu serves as analysis tool and can:

- display the number of programmed ward bus users,
- display the active time zone,
- test group lamp, fault message lamp and fault message relay,
- simulate a mains failure message.

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## Operation of the service menu

<b>Starting the service menu</b>	Keep <b>buttons 1 and 2</b> pressed simultaneously for 10 seconds. The software version of the group controller is displayed. The service menu is active. The LEDs display the same information as during normal operation, see the table above.
<b>Scrolling forward</b>	Press <b>button 2</b> .
<b>Scrolling back</b>	Press <b>button 1</b> .
<b>Quitting the service menu</b>	Keep <b>buttons 1 and 2</b> pressed simultaneously for 3 seconds.
<b>Note:</b> If no button is pressed within a period of 60 seconds, the service menu is terminated automatically. Settings already made are not saved.	

## Changing the language of the service menu

1. Scroll to "*Language*" in the service menu.
2. Press **Button 4** to select "*Language*".
3. Keep pressing **Button 1** until the desired language appears.
4. Press **Button 2** to apply the new setting. The setting is saved only after quitting the service menu.

## Displaying the number of programmed ward bus subscribers

1. Scroll to "*Ward-Dev*" in the service menu.
- ✓ The number of ward bus users programmed in the group controller is displayed. This number must match the number of existing ward bus users.
2. If the number displayed does not match the number of existing ward bus users, correct the programming with the management software.

## Displaying the active time zone

1. Scroll to "*Time zone*" in the service menu.
- ✓ The active time zone is displayed.

## Testing the function of the fault message relay

1. Scroll to "*Test Relais*" in the service menu.
2. Press **Button 4**.
- ✓ The relay is switched over. The relay can be toggled as often as desired by pressing **Button 4**.
3. To ensure the correct status of the relay after the test, the group controller must be reset. To do this, keep **Buttons 5 and 6** pressed for 5 seconds.

## Testing group lamp

1. Scroll to "*Test Gr-Lamp*" in the service menu.
2. Press **Button 4**.
- ✓ The connected group lamp lights up for 30 seconds.

## Testing the fault message lamp

1. Scroll to "*Test Er-Lamp*" in the service menu.
2. Press **Button 4**.
- ✓ The connected fault message lamp lights up for 30 seconds.

## Simulating a mains failure message

1. Scroll to "*Simulate MainFail*" in the service menu.
2. Press **Button 4**.
- ✓ A mains failure message is displayed in the nurse call system (fault) for 30 seconds.