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ePat[®]lite spare parts set connection cable, order no. 77 0383 00, 77 0383 10

Spare parts set to replace the connection cable of the ePat[®]lite.
ATTENTION! You will need a drop of super glue gel. This is not included in the spare parts set 77 0383 10.

Replacing the connection cable

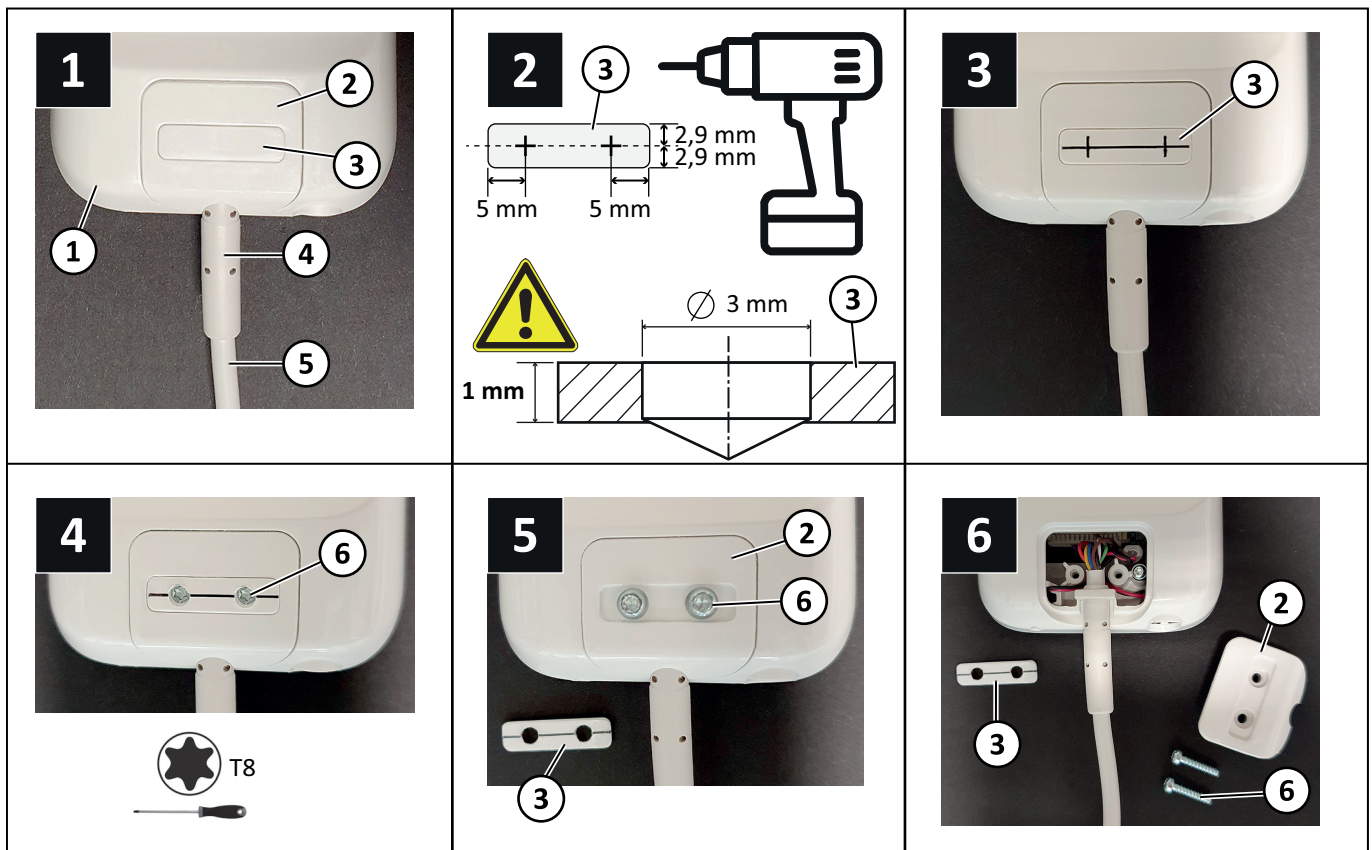
To replace the connection cable [5], the service cover [2] must be removed, which is fixed with 2 screws [6]. The screws [6] are covered by a cover plate [3], which is fixed with a glue dot in the middle.

1. Mark the position of the two screws on the cover plate [3] according to Fig. 2 and 3.

ATTENTION! Do not drill into the thread of the screws.

2. Carefully drill only 1 mm deep with a 3 mm HSS drill bit at the marked positions until the screws [6] are visible, see Fig. 4.
3. Loosen the screws [6] with a T8 Torx screwdriver, Fig. 4. When doing so, lever off the stuck cover plate [3], Fig. 5.
4. Remove the service cover [2], see Fig. 6.
5. Memorise how the cable grommet [4] of the connection cable is inserted in the housing and remove it, Fig. 6.

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EN - Installation Instructions

- Carefully disconnect the plug-in connector [7] from the socket [8], see Fig. 7.
- Carefully connect the plug-in connector [9] of the new connection cable into the socket [8]. Observe the orientation of the connector [9]: The unused pins are on the left side, see Fig. 8.
- Carefully press with a non-conductive object until the plug connection is tight, see Fig. 8.
- Insert the cable grommet [10]. Make sure that no wire is jammed and that the two screw holders [14] are not covered, Fig. 8 and 9.
- Fix the service cover [2] with the two screws [6], Torx screwdriver T8, torque value: 0.5 Nm, Fig. 11.
- Perform a functional test of the ePat[®]lite.
ATTENTION! Follow the application instructions enclosed with the super glue gel.
- Place a drop of the super glue gel between the two screws and place the cover accurately on top, Fig. 12.
- Allow the glue joint to harden.

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