

**Warning**




The use of a call device does not release supervisors from the obligation to supervise the persons supported by such a device. The call device provides for supporting comfort and allows for checking supervised persons less frequently. The system must be subjected to regular functional checks. Please note that the reach of radio links can be subject to substantial variations especially in buildings and that the product will not work in case of power failure. In the case of error and malfunction the system must not be used until it has been checked by an authorised technician. Carefully read all attached instructions before operating this device and keep them for further reference.

Important: This device must not be used for monitoring purposes if life-threatening conditions are foreseeable. In addition, control measures for checking the quality of the radio link are to be observed if the device uses a radio link.

**Note**

This quick reference guide describes, in a simplified form, the operation of a completely connected and adjusted **VarioSwitch** breath sensor device. Please refer to the instruction manual LE218 for connection, placing into operation and comprehensive operation instructions.

**Fig. 1: placement and controls**

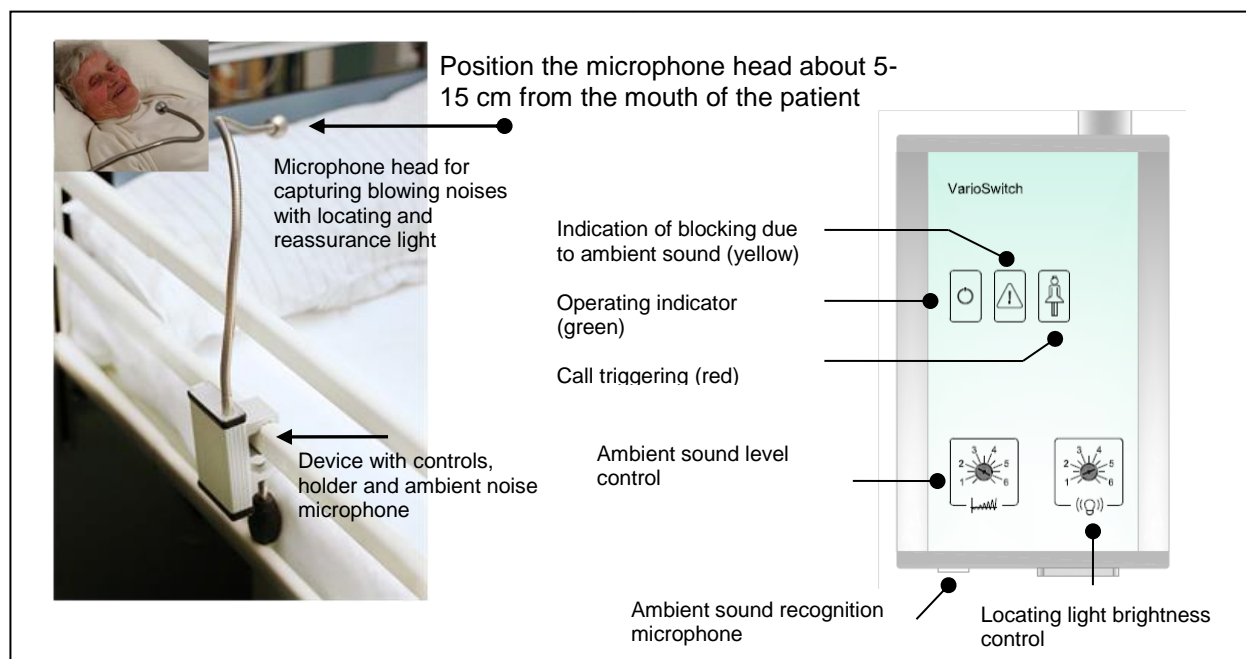

**Functions**

The gooseneck microphone head of the **VarioSwitch** breath sensor captures blowing noises intended as calls by the patient and hence triggers a patient call. The second microphone at the bottom side detects ambient sound. As long as the ambient sound is too loud, triggering of calls is blocked in order to avoid faulty activations by loud background noise. The three operating indicators indicate the device status.

**Table: Functions and Operating Indicators**

Indicator element/control	Icon	Operating state
Operating indicator (green)		Must be permanently lit
Call indicators (red)		Light up during a call and go out afterwards.
Ambient sound (yellow)		Lights up in the case of loud ambient noise. Calls cannot be triggered as long as this indicator lights up.
Locating light (at the microphone head) (blue)		Lights up permanently (brightness can be adjusted)

**Switch off call message**

The **VarioSwitch** breath sensor sends a call pulse to the call system. The call message is switched off at the call device.