



Warning

The use of a call device does not release supervisors from the obligation to supervise the persons supported by such a device. This system is radio-based and must therefore not be used for monitoring purposes if life-threatening conditions are foreseeable. This quick reference guide does not release the user from the obligation to read and understand the contents of the complete **VarioRec6** manual LE243, available on www.lehmannweb.de

Note: In this quick reference guide the "VarioRec6 radio receiver" is referred to as "radio receiver" or "receiver".

This quick reference guide describes only the basic functions. It does not cover failures.

Starting the Radio Receiver and Receiving a Radio Call

1 unitPlug in receiver

Power Supply

2 Receiver is operational
Green

3 Trigger radio call
(transmitters already programmed)

4 Receiver indicates and transmits call
1.5 s red

Programming a Radio Transmitter

Before a radio receiver can detect a radio transmitter, the latter has to be programmed in the radio receiver.

1 Press and hold the right-hand button when plugging

Power Supply unit*

2 Receiver is in programming mode
Green flashing light

3 Trigger radio call

4 New radio transmitter has been detected
1.5 s red

5 Briefly press right-hand button

6 The new radio transmitter has been programmed
1.5 s green

7 Press and hold the left-hand button

8 Receiver is operational again
Green

*Press the right button while connecting the power supply unit.

Deleting a Radio Transmitter

A radio receiver ceases to detect a radio transmitter only after the latter has been deleted from the radio receiver.

1 Press and hold the right-hand button when plugging

2 Receiver is in programming mode
Green flashing light

3 Trigger a radio call at the radio transmitter to be deleted

4 Radio transmitter has been detected
Green / red flashing light

5 Briefly press right-hand button

6 Radio transmitter has been deleted
1.5 s green

7 Press and hold the left-hand button

8 Receiver is operational again
Green

Radio Receiver Messages

Important: These messages occur only in the operating state and indicate a fault / error. Acknowledge deletion of the message. Please consult the complete manual LE 243 for detailed information on the respective message.

Message	Flash sequence	Measure	Acknowledge signal message
Transmitter battery low	Red flashing light	Change radio transmitter battery	Briefly press left- or right-hand button
No daily message	Green flashing	Check radio transmitter	
Vital message missing	Green/orange flashing	Looking for reason of missing message	The message has been deleted, receiver is operational again grün
Reception blockage	Orange flashing	Check for sources of radio	
System error	Green/red flashing	Error in radio receiver or in the connection	
Static call	Static light	Turn back the receiver	